

Welch Allyn Connex® ProBP 3400

Maintenance and Cleaning

Calibrate the device

Welch Allyn recommends annual calibration of the ProBP 3400.

To calibrate the ProBP 3400,

- o Use the Welch Allyn Service Tool.
- o Send the device to a Welch Allyn Service Center near you.

Change the battery

The ProBP 3400 device is powered by a lithium-ion rechargeable battery.

To remove the battery:

1. Remove the screw from the battery compartment door with a Phillips screwdriver.
2. Remove the battery compartment door.
3. Remove the battery from the battery compartment.
4. Remove the new battery from the shipping bag.
5. Using the polarized guidelines on the battery label, insert the battery, positive (+) side first, into the battery compartment.
6. Replace the battery compartment door.
7. Secure the screw with the screwdriver.

If the ProBP 3400 will be unused for a long period of time, remove the battery prior to storing the device.

Clean the device

The following agents are compatible with the device:

- o CaviWipes™
- o Sani-Cloth®Plus
- o 70 percent isopropyl alcohol
- o 10 percent chlorine bleach/90 percent water solution

Note Clean the device on a routine basis, according to your facility's protocols and standards or local regulations.

CaviWipes or Sani-Cloth Plus

Follow the cleaning agent manufacturer's guidelines.

1. Using CaviWipes or Sani-Cloth Plus, wipe the surface of the device to remove all debris.
2. Allow the device surface to dry for a minimum of 10 minutes before using the device.

70 percent isopropyl alcohol

Wipe the device with a clean cloth slightly dampened with 70 percent isopropyl alcohol.

10 percent chlorine bleach/90 percent water solution

1. Wipe the device with a clean cloth slightly dampened with a 10 percent bleach and 90 percent water solution.

2. Rinse with a clean cloth slightly dampened with clean water.

46 Maintenance and service ProBP 3400

3. Allow the device surface to dry for a minimum of 10 minutes before using the device.

Clean the accessories

Wipe the NIBP hose and any reusable cuffs with a damp cloth moistened in a mild detergent solution.

The same cleaning agents used to clean the device can be used on the stand and mounting accessories.

Note Clean the device on a routine basis, according to your facility's protocols and standards or local regulations.