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It is my pleasure to introduce to you the new 2017 KaWe product catalogue.

The demand for medical devices is continually growing. In the healthcare system worldwide there will be many large investments made over the next few years that will have a positive effect on our future.

There is a special interest in high-quality, innovative and easy-to-use devices. This trend is clearly the focus of KaWe's product development. Therefore, we will continue to market products that meet these requirements. This is also reflected by our latest product innovations.

Our KaWe MASTERLIGHT® LED Focus, which is a new addition to our line of examination lights, and our KaWe EUROLIGHT® Pro-Set LED, which is the perfect solution for the ENT specialist.

Another very important aspect is the ever-increasing hygienic demands in healthcare. KaWe also offers innovative and cost-effective solutions with our single-use products.

In order to meet all these important requirements, KaWe will be building a new modern Logistics and Production Centre in 2017. Our strict quality management guarantees you a consistently high level of product quality now and in the future. The entire KaWe team is at your disposal at any time with its expertise and first-class service.

KaWe has been offering high-quality medical products worldwide for more than 125 years. I am proud of this and it is an honor for me to successfully continue this long family tradition into its 4th generation.

I wish you every possible success and hope you enjoy working with the new KaWe catalogue and thank you for your trust. It is a pleasure working with you.

Yours truly, Regina Kirchner-Gottschalk  
Managing Director

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# Table of contents

1	Overview of product groups				
2	Table of contents				
4	ISO-Certificates				
5	Innovations				
6	KaWe history				
<b>8</b>	<b>Otoscopies</b>				
9	Overview				
10	KaWe EUROLIGHT®/ KaWe COMBILIGHT® Detailed view				
12	KaWe EUROLIGHT®				
15	KaWe COMBILIGHT®				
18	KaWe PICCOLIGHT®				
22	Disposable ear funnels				
23	Reusable ear funnels				
24	KaWe EUROLIGHT® / KaWe COMBILIGHT® heads				
25	KaWe EUROLIGHT® / KaWe COMBILIGHT® handles				
26	KaWe PICCOLIGHT® heads and handles				
26	Bags for otoscopes				
27	Accessories				
<b>28</b>	<b>Ophthalmoscopes</b>				
29	Overview				
30	KaWe EUROLIGHT® KaWe COMBILIGHT® Detailed view				
31	KaWe PICCOLIGHT® Detailed view				
32	KaWe EUROLIGHT®				
34	KaWe PICCOLIGHT®				
36	KaWe EUROLIGHT® heads				
37	KaWe EUROLIGHT® handles				
38	KaWe PICCOLIGHT® heads and handles				
38	Bags				
39	Apertures				
<b>40</b>	<b>Dermatoscopes</b>				
41	Overview				
42	KaWe EUROLIGHT® Detailed view				
43	KaWe PICCOLIGHT® Detailed view				
44	KaWe EUROLIGHT®				
44	KaWe PICCOLIGHT®				
45	KaWe EUROLIGHT® KaWe PICCOLIGHT® heads and handles				
45	Spare parts				
<b>46</b>	<b>KaWe MedCenter® 5000 diagnostic-sets</b>				
47	Features				
48	KaWe EUROLIGHT® E25 Detailed view				
49	KaWe MedCenter® 5000 sets				
51	KaWe MedCenter® 5000 modules				
52	KaWe MedCenter® 5000 heads				
53	Overview of diagnostic sets				
54	Basic sets				
58	KaWe EUROLIGHT® sets				
61	KaWe COMBILIGHT® sets				
68	KaWe PICCOLIGHT® sets				
73	Accessories				
<b>74</b>	<b>F.O. Laryngoskope</b>				
75	F.O. handle detailed view				
76	F.O. Xenon				
77	F.O. LED				
78	F.O. Xenon battery and charging handles				
79	F.O. Xenon charging handles				
80	F.O. LED battery handles				
81	F.O. LED battery/charging handles				
82	F.O. blades				
83	Disposable blades				
84	KaWe MEGALIGHT® Macintosh F.O.				
85	Macintosh F.O.				
86	TEPRO Macintosh F.O.				
87	KaWe FLAPLIGHT® F.O./ KaWe FLAPLIGHT® F.O. ME- GALIGHT®				
88	KaWe MEGALIGHT® Miller F.O.				
89	Miller F.O.				
90	Foregger F.O./ Polio Macintosh F.O.				
91	Economy Macintosh F.O.				
92	Economy Miller F.O.				
93	Macintosh disposable blades, metal				
94	Miller disposable blades, metal				
95	Macintosh/Miller disposable blades, plastic				
96	F.O. Laryngoscope sets				
<b>102</b>	<b>Laryngoskope C</b>				
103	Detailed view				
104	C handles				
105	C blades, disposable blades				
106	Macintosh C				
107	Miller C				
108	Foregger C				
109	KaWe Flaplight® C/ Polio Macintosh C				
110	Economy Macintosh C				
111	Economy Miller C				
112	Macintosh C disposable blades, metal				
113	Miller C disposable blades, metal				
114	C laryngoscope sets				
117	Laryngoscope cases				
<b>120</b>	<b>Blood pressure measuring devices</b>				
121	Detailed view				
120	Aneroid blood pressure measuring devices A1, A1+				
121	Aneroid blood pressure measuring devices A2, A2+				
122	KaWe MASTERMED® C Table/wall-mounted/stand type, blood pressure measu- ring device				
<b>124</b>	<b>SwiSto3 Iontophoresis device</b>				
125	Detailed view, features				
126	Product description				
127	Iontophoresis set				
129	Special accessories/spare parts				
<b>130</b>	<b>Stethoscopes</b>				
132	Overview of stethoscopes				
	<b>Twin-head stethoscopes</b>				
133	Planet				
134	Profi Cardiology				
135	Standard Prestige				
136	Standard Prestige light				
137	Kirchner Colorscop® Duo				
138	Double-stethoscope				
	<b>Flat head stethoscope</b>				
139	Top-Cardiology				
140	Kirchner Colorscop® Plano				
141	Single-stethoscope				
	<b>Bell type stethoscopes</b>				
142	Petiphon stethoscope for paediatrics				
143	Suprabel stethoscope for veterinarians				
144	Training stethoscope Plano/Duo				
145	Pinard stethoscope				
146	Stethoscope accessories				
<b>148</b>	<b>Reflex hammers</b>				
149	Colorflex				
150	Babinsky				
150	Rossier				
151	Varioflex				
152	Trömner				
153	Witroe				
153	Buck				
<b>154</b>	<b>Tuning Forks</b>				
155	Rydel Seiffer tuning fork				
156	Lucae tuning forks				
157	Aluminium tuning forks				
158	Hartmann tuning forks				
159	Tuning fork set				
<b>160</b>	<b>Medical equipment</b>				
161	Infusion stand made of stainless steel				
162	Infusion stand, heavy				
163	Infusion stand, chrome-plated				
164	Examination couch Classic				
165	Swivel stool Classic Foot stool				
<b>166</b>	<b>Lighting</b>				
167	KaWe MASTERLIGHT® HL				
168	KaWe MASTERLIGHT® LED				
169	Accessories				
	<b>KaWe MASTERLIGHT® Examinationslamps LED</b>				
170	7W				
171	10W				
172	18W / 18W focus				
173	20W / 20W focus				
<b>174</b>	<b>Metal instruments</b>				
175	All-metal syringes				
176	Tick-Fix tick tweezers All-purpose bandage scissors				
177	Lister scissor Pea clamp				
178	Cusco vaginal specula				
179	Laryngeal mirrors				
<b>180</b>	<b>Various articles</b>				
182	KaWe HiLight® LED H-800				
183	Sets				
184	Accessories, spare parts				
184	Standard forehead mirror according to Clar				
185	Cliplight diagnostic pen lights				
185	Diagnostic light, white,				
185	KaWe DIALIGHT® XL diagnostic pen light				
186	KaWe easy clic tourniquet				
186	Stripp-Quick tourniquet loop				
186	Liquimed swab moistener				
186	Liquifix swab moistener				
186	Silicone spray can				
187	KaWe PERSON CHECK® height measuring device				
187	Measuring tapes				
187	EKG ruler				
188	Goniometer				
188	Guedel airway tubes				
188	Disettes				
189	KaWe MedCharge® 4000 charging station				
189	Rechargeable batteries				
190	Bulbs				
<b>192</b>	<b>Binocular magnifying glasses</b>				
193	Detailed view				
194	BC 2.3				
195	BK 2.3				
195	BS 2.3				
<b>196</b>	<b>Thermometers</b>				
197	KaWe Temp® 10S				
197	KaWe Temp® 20F				
198	KaWe Temp® 30IR				
	General Terms and Conditions of Sale				

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Our products meet product-specific standards and norms and are continually being improved upon in order to keep them up-to-date and in line with the most modern technology. For the safety of patients, users and third parties, our products are subject to a risk management process during all phases of the product life cycle in accordance with standard DIN EN ISO 14971. Naturally, our reprocessable products are compliant with the standard EN ISO 17664.

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Our medical products are known for their high quality and exceptional reliability. The EN ISO 9001 certification with EN ISO 13485 and the CE label serve to assure you that with each purchase of a Ka-We product, you are receiving a top quality instrument.

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Our KaWe brand, technical processes and our designs are and have always been patent protected.

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Our primary goal is to protect our authorized dealers as well as users and patients from plagiarized products and the consequences thereof. Plagiarism results in reputation damage for all concerned, and in addition, such articles do not comply with Medical Devices Act requirements, which define the safety, suitability and performance of medical devices and protect the patient, user and third parties.

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When you purchase original KaWe products, you can be sure that we as the manufacturer have used the suitable conformity evaluation process to meet essential health and safety requirements.

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Certificates and annex certificates are available in our download center at [www.kawemed.de](http://www.kawemed.de).

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Certificates and annex certificates can be found in our new download center at [www.kawemed.de](http://www.kawemed.de)

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**2 year warranty on all products,**  
**except bulbs, rechargeable batteries (Chapter 12), batteries**  
**4 year warranty on SwiSto3 (Chapter 5)**  
**5 year warranty on LEDs (Chapter 1-3)**

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## Time for innovation – outlook

Over the last 125 years, KaWe has grown into a modern, globally operating medical technology enterprise. Research and development are top priority at KaWe.

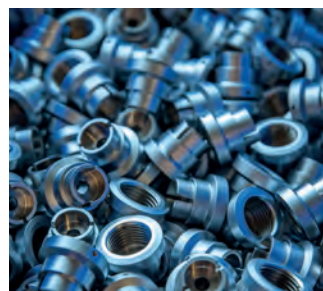
Over the past few years, the Research and Development department has been expanded and personnel has been increased. Under the guidance of a medical consultant, new and innovative products are being developed. One of the things that is extremely important to KaWe during the new product development phase is customer feedback.

The results of this cooperation are our newest products for thermometry, blood-pressure monitoring, examination lights and ophthalmology.

In the future, the focus of our Development department will be, above all, on electronic and optical products. For all new developments, precision and high-quality are always at the forefront.

This Regina Kirchner-Gottschalk also knows: Innovations are the guarantors of the future.

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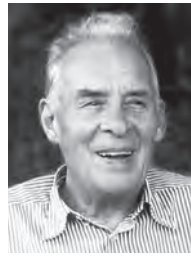
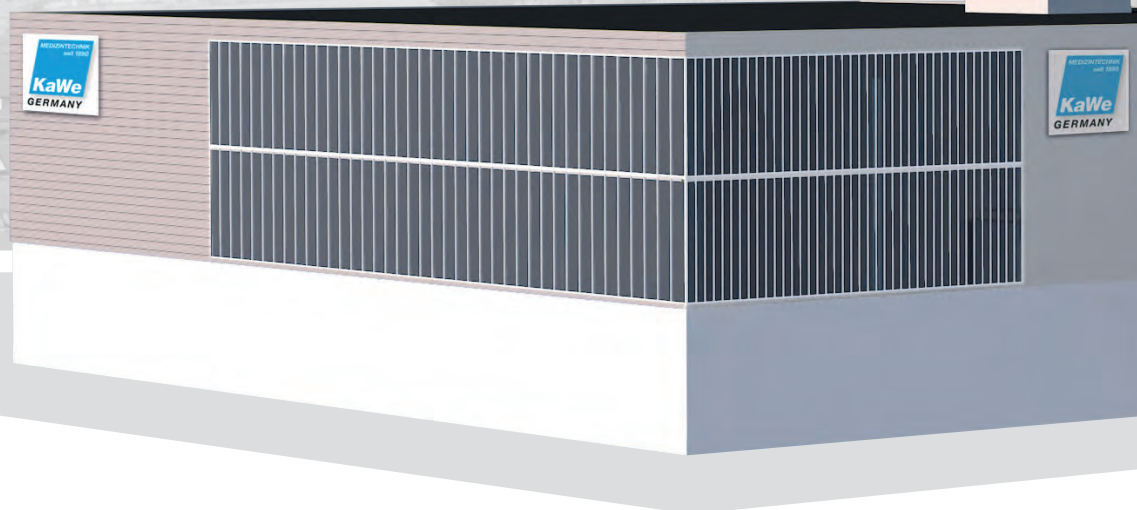
Regina Kirchner-Gottschalk

7

KIRCHNER & WILHELM GmbH & Co.KG was founded on 1st April 1890 in Stuttgart by Gustav Kirchner and Hugo Wilhelm. Medical instruments for diagnostic and surgical procedures were already being manufactured at that time for the entire German market.

Toward the end of 1916, due to his failing health, Hugo Wilhelm turned the business completely over to Gustav Kirchner.

Two years later, in 1918, his eldest son Albert joined the executive board which he headed until 1962.

In 1962, his son Georg Kirchner assumed the position of sole managing director and directed the fate of the company for the rest of his life.

Regina Kirchner-Gottschalk joined the family company in Stuttgart in 1986 as a member of upper management and then in 1997 became the executive director. Since then she has steadily expanded foreign operations and raised export numbers by 70%.

Now the company is represented globally in more than 100 countries. In 1989, the company moved to its current location in Asperg (district of Ludwigsburg.)

Due to this expansion, the newly-constructed building soon became too small.

The addition of an additional floor in 2008 increased the area of the building by about a third.

The year 2017 will be another milestone in the history of the company. Another expansion of our commercial premises will be carried out. In order to optimally meet today's demands, a new, large and modern Logistics and Production Centre will be built behind our current building. This will enable KaWe to be optimally prepared for the future and allow it to respond to customer's requests with the required flexibility.

KaWe is surging forward with strong entrepreneurial vision and Swabian practicality. From Regina Kirchner-Gottschalk's perspective, KaWe is optimally positioned for the future and for continued corporate expansion.

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## LED-lights continue to be a triumphant success

The development of LED technology has paved the way for many new possibilities in the field of instrument engineering. Experts even say that there has been a paradigm shift from conventional lights to LED lights because LED technology has developed so astonishingly in the past few years. Most notably, the luminous flux was increased in relation to the power input. The advantages of LED technology over conventional illumination are a longer lifespan and a lower energy consumption of the light emitting diodes. Improvements in the resilience (resistance to impact and vibration) have also played a major role.

LED illuminants reach their maximum illumination intensity immediately. The light does not flicker, preventing the occurrence of the strobe effect. In addition, LED lights do not heat up the area that is being illuminated. This makes medical equipment even safer. The LED technology used in the KaWe EUROLIGHT®, KaWe COMBILIGHT® 2.5V, 3.5V and KaWe PICCOLIGHT® F.O. Otoscopes ensures superb lighting efficiency and excellent light transmission and makes highly-functional illumination possible at a low price.

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	KaWe EUROLIGHT®						KaWe COMBILLIGHT®						KaWe PICCOLIGHT®							
	C10	C30	C30OP LED/s	C30VET	C30VET OP	F.O.30	F.O.30 LED/s	F.O.30	F.O.30	F.O.30 LED/1hp	C10	F.O.30	F.O.30 LED/s	F.O.30	F.O.30	F.O.30 LED/1hp	C	F.O.	F.O. LED/s	F.O. LED/1hp
Versorgungsspannung [V]   supply voltage [V]	2,5	2,5	2,5	2,5	2,5	2,5	2,5	3,5	3,5	3,5	2,5	2,5	2,5	3,5	3,5	3,5	2,5	2,5	2,5	2,5
Ladespannung [V]   charge voltage [V]									240V						240V					
Stecker   plug									EU						EU					
Beleuchtungsart type of illumination	F.O.					•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	C	•	•		•	•					•						•			
	LED			•																
	F.O. LED standard							•					•						•	
	F.O. LED high power									•						•				•
Kopf   head	Metallkopf   metal head	•	•	•	•	•	•	•	•	•										
	Kunststoffkopf   plastic head										•	•	•	•	•	•	•	•	•	•
Verschluss fastening mechanism	Clic-Verschluss   click-on connection		•	•	•	•	•	•	•	•	•	•	•	•	•	•				
	Verschluss mit Schraube   connection with fastening screw	•									•									
	Schraubverschluss   screw-on connection																•	•	•	•
Griff   handle	Metallgriff   metal handle	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•				
	Kunststoffgriff   plastic handle																•	•	•	•
Lupe schwenkbar pivoting magnifying glasses	mit 3-facher Vergrößerung   with 3-times magnification	•	•			•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	mit 2,5-facher Vergrößerung   with 2.5-times magnification			•																
	mit 2-facher Vergrößerung   with 2-times magnification				•	•														
Leuchtmittel illuminant	Xenon-Lampe   xenon bulb					•	•	•			•	•	•	•			•			
	Vakuum-Lampe   vacuum bulb	•	•		•	•					•						•			
	LED standard-Lampe   LED standard bulb			•			•						•						•	
	LED high power-Lampe   LED high-power bulb									•						•				•
Lebensdauer Leuchtmittel service life of illuminant	ca. 15 Std.   approx. 15 h	•	•		•	•					•						•			
	ca. 20 Std.   approx. 20 h						•	•	•			•	•	•				•		
	ca. 50.000 Std.   approx. 50,000 h			•			•			•						•				•
Lichtregler light regulator	•	•		•	•	•		•	•		•	•		•	•					
aufladbar in rechargeable in	Ladestation KaWe MedCharge® 4000 (mit KaWe Ladebatterie) KaWe MedCharge® 4000 charging station (with KaWe rechargeable battery)	•	•		•	•	•	•	•		•	•		•	•					
	Steckdose   electric outlet								•						•					
mögliche Energie- quelle   energy source options	Batterie(n)   battery (batteries)	•	•	•	•	•	•				•	•	•				•	•	•	•
	KaWe-Ladebatterie   KaWe rechargeable battery	•	•		•	•	•	•	•	•	•	•		•	•	•				
im Lieferumfang enthalten included items	20 Einweg-Ohrtrichter   20 disposable ear funnels (Ø 2,5/4,0 mm)	•	•			•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
	3 Dauer-Ohrtrichter   3 reusable ear funnels (Ø 2,5/3,5/4,5 mm)										•									
	3 Dauer-Ohrtrichter VET   3 reusable ear funnels (VET) (Ø 4,0/5,0/7,0 mm)				•	•														
	5 Dauer-Ohrtrichter   5 reusable ear funnels (Ø 2,5/3,5/4,5/5,5/9,0 mm)			•																
	Ladebatterie 3,5 V   rechargeable battery 3.5 V							•	•	•				•	•	•				
	Reißverschlusstasche   bag with zipper	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•				
	Stofftasche   cloth bag																•	•	•	•
erhältlich in available in																	•	•	•	•

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### Metal handle – dual mode

- ① with rechargeable battery = **charging handle**  
for KaWe MedCharge® 4000 charging station
- ② with batteries = **battery handle**

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- 1 3-fold pivoting  
magnifying glass
- 2 Ear funnel
- 3 Connection for  
pneumatical test
- 4 Click-on connection
- 5 Lamp (not visible)
- 6 Push-button switch
- 7 Light regulator,  
continuously variable  
dimmer  
(except for LED)
- 8 Batteries
- 9 Rechargeable battery
- 10 Battery/rechargeable  
battery charging  
handle
- 11 Metal cap
- 12 End cap

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## Made by professionals for professionals

The high-quality metal housing of the KaWe EUROLIGHT® product line makes the compact instruments sturdy – they are actually practically indestructible. With this instrument you are not only choosing excellent quality but also high durability. Your long-lasting companion for ear, nose and throat diagnostics, is also available with high-quality distal fibre-optics (F.O.) in 2.5V or 3.5 V in connection with a high-capacity xenon light or with standard illumination C and guarantees a bright illumination free of reflection of the tympanum and auditory canal. The light strength can be easily regulated with the rheostat (except for the LED). A further advantageous feature is the pivoting magnifying glass with 2, 2.5 or 3-fold magnification. The KaWe EUROLIGHT® is also available with automatic click-on or screw-on connection.

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#### Charging station

complete with power supply  
unit and universal power plug  
for Lithium-Ion technology

EU / GB / US

# KaWe EUROLIGHT®

## PRODUKTSERIE | PRODUCT LINE

12

### KaWe EUROLIGHT® Otokoskope | Otoroscopes

2,5 V

1	C10	VL	-	
2	C30	VL	-	
3	C30 OP LED	LED	-	-
4	VET C30	VL	-	
5	VET C30 OP	VL	-	
6	F.O.30	XL	-	
7	F.O.30 LED	LED	-	-

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#### Features:

- optimal light intensity (centric)
- **2.5 V metal handle with dual mode** (except for charging handle for electric outlet and LED)
- Battery handle:** to be used with 2 batteries (type C, 1.5V)
- Charging handle:** suitable for rechargeable batteries (not included with the KaWe EUROLIGHT® 2.5 V)
- pivoting magnifying glass with 2-fold magnification VET C30, C30 OP, 2.5-fold C30 OP LED and/or 3-fold magnification
- blower connection (except for C30 OP, C30 OP LED)

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Ladebatterie im Lieferumfang enthalten / rechargeable battery is included

VL Vakuum-Lampe / vacuum bulb  
LED LED-Lampe / LED bulb

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VL Vakuum-Lampe / vacuum bulb  
XL Xenon-Lampe / xenon bulb  
LED LED-Lampe / LED bulb

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- click-on connection
- rechargeable in electric outlet (EU 240 V)
- high-power 3.5V xenon bulb
- metal charging handle C
- dimmable rheostat
- disposable ear funnels (»22)
  - 10 x Ø 2.5 mm, 10 x Ø 4.0 mm
- includes 3.5V NiMH rechargeable battery
- in bag with zipper

**9** F.O.30 | 3,5V | XL | EU

REF 01.11430.811

Leuchtmittel | illuminant

REF 12.75241.003

aufladbar in Ladestation KaWe MedCharge® 4000 in Verbindung mit Ladebatterie  
rechargeable in KaWe MedCharge® 4000 charging station with rechargeable battery

Ladebatterie im Lieferumfang enthalten / rechargeable battery is included

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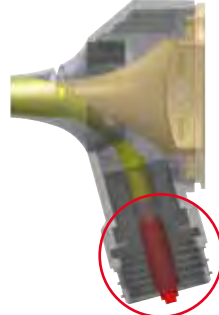
## Bring light into the darkness

Technologically-mature and tested for years, here KaWe is also committed to excellence in quality and combines high-quality fibre optics with high-power 2.5V or 3.5V xenon light for especially bright and white light. It goes without saying that the selection of proven, standard KaWe COMBILIGHT® otoscopes is also an excellent choice. The KaWe COMBILIGHT® optimally illuminates the auditory canal and the eardrum and helps you make an accurate diagnosis. The housing is robust and they operate reliably. Otoscopes from the KaWe COMBILIGHT® product line have an excellent cost/performance ratio.

Ladestation  
Charging station  
KaWe MedCharge® 4000

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### Features:

- optimal light intensity (centric)
- **metal handle 2.5 V with dual mode** (except charging handle for electric outlet and LED)
- Battery handle:** to be used with 2 C batteries (1.5V)
- Charging handle:** suitable for rechargeable battery (not included with the KaWe COMBILIGHT® 2.5 V)
- pivoting magnifying glass with 3-fold magnification and/or 2-fold magnification C10
- blower connection

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- fastening screw connection
- vacuum bulb
- metal battery/charging handle C
- dimmable rheostat
- 3 reusable ear funnels (»23) Ø 2.5 / 3.5 / 4.5 mm
- in bag with zipper
- batteries: 2 C batteries

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- click-on connection
- Xenon bulb
- metal battery/charging handle C
- dimmable rheostat
- disposable ear funnels (»22) 10 x Ø 2.5 mm 10 x Ø 4.0 mm
- in bag with zipper
- batteries: 2 C batteries

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- click-on connection
- LED bulb
- illuminance approx. 4,200 Lux (at a distance of 15 mm)
- life of bulb approx. 50,000 h
- metal battery handle C
- disposable ear funnels (»22) 10 x Ø 2.5 mm, 10 x Ø 4.0 mm
- in bag with zipper
- batteries: 2 C batteries

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- click-on connection
- high-power 3.5 V xenon bulb
- metal charging handle C
- dimmable rheostat
- disposable ear funnels (»22)  
10 x Ø 2.5 mm, 10 x Ø 4.0 mm
- incl. 3.5 V Li-ion rechargeable battery (only to be used with KaWe MedCharge® 4000 charging station (»189)
- in bag with zipper

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- click-on connection / LED bulb
- illuminance approx. 17,000 Lux (at a distance of 15 mm)
- life of bulb approx. 50,000 h
- metal charging handle C
- disposable ear funnels (»22)  
10 x Ø 2.5 mm, 10 x Ø 4.0 mm
- incl. 3.5 V Li-ion rechargeable battery (only to be used with KaWe MedCharge® 4000 charging station (»189)
- in bag with zipper

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- click-on connection
- rechargeable in electric outlet (EU 240 V)
- high-power 3.5 V xenon bulb
- metal charging handle C
- dimmable rheostat
- disposable ear funnels (»22)  
10 x Ø 2.5 mm, 10 x Ø 4.0 mm
- includes 3.5V NiMH rechargeable battery
- in bag with zipper

6 F.O.30 | 3,5V | XL | EU

REF 01.12430.811

Leuchtmittel | illuminant

REF 12.75241.003

aufladbar in Ladestation KaWe MedCharge® 4000 in Verbindung mit Ladebatterie / rechargeable in KaWe MedCharge® 4000 charging station with rechargeable battery  
 Ladebatterie im Lieferumfang enthalten / rechargeable battery is included

**XL** Xenon-Lampe / xenon bulb  
**LED** LED-Lampe / LED bulb

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- 1 3-fold pivoting  
magnifying glass
- 2 Ear funnel
- 3 Connection for  
pneumatical test
- 4 Bulb (not visible)
- 5 Threaded screw-on  
connection
- 6 On/off sliding switch
- 7 Metal clip
- 8 Battery handle
- 9 Batteries

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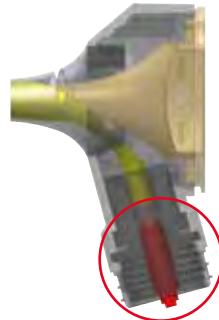
#### Features:

- variety of illumination options
- pocket-size, light-weight, handy, modern design
- optimal light strength (centric)
- ergonomic head shape
- pivoting magnifying glass with 3-fold magnification
- integrated clip shut-off function
- blower connection

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#### night. sky. stone. – A new focus

With the trendy new colours: night, sky and stone, we are initiating an elegant new design era. Trendsetting LED technology completes the KaWe PICCOLIGHT® line perfectly.

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21

The proven LED KaWe PICCOLIGHT® F.O. otoscopes provide you with optimal conditions for easy ENT treatment through high-quality light technology. The LED illuminant has an excellent light efficiency and very high light transmission. LED illuminants are sure to please with their long working lives and very robust design. The xenon illumination of KaWe PICCOLIGHT® F.O. otoscopes that you already own can be converted to LED-illumination simply by replacing the bulb. (»190-191)

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- F.O. LED illumination
- illuminance approx. 4,200 Lux (at a distance of 15 mm)
- life of LED approx. 50,000 h
- shockproof and energy-saving
- battery handle AA
- disposable ear funnels (»22)  
10 x Ø 2.5 mm  
10 x Ø 4.0 mm
- in cloth bag
- battery: 2 Mignon (type AA) ☒

3	F.O. LED   2,5 V   LED   night	■	REF 01.13500.021
	F.O. LED   2,5 V   LED   sky	■	REF 01.13500.232
	F.O. LED   2,5 V   LED   stone	■	REF 01.13500.262

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- F.O. LED high power illumination
- illuminance approx. 18,400 Lux (at a distance of 15 mm)
- life of bulb approx. 50,000 h
- shockproof and energy-saving
- battery handle AA
- disposable ear funnels (»22)  
10 x Ø 2.5 mm  
10 x Ø 4.0 mm
- in cloth bag
- battery: 2 Mignon (type AA) ☒

4	F.O. LED hp   2,5 V   LED   night	■	REF 01.13600.022
	F.O. LED hp   2,5 V   LED   sky	■	REF 01.13600.232
	F.O. LED hp   2,5 V   LED   stone	■	REF 01.13600.262

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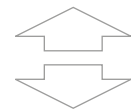
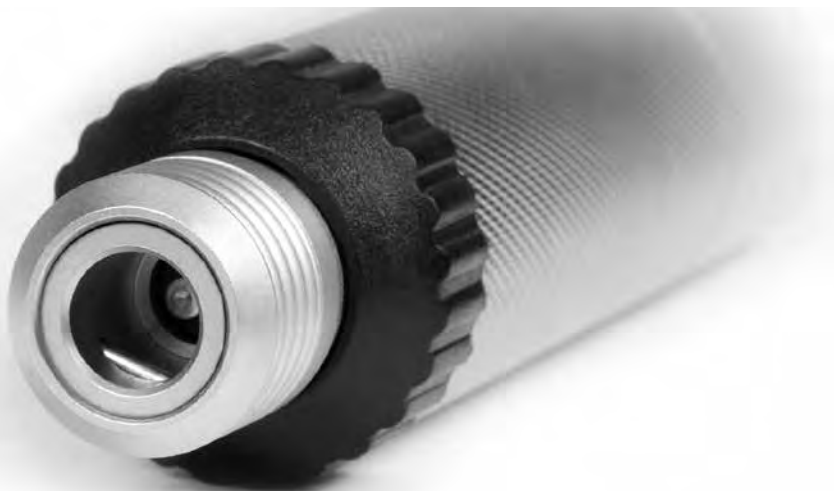
Rechargeable battery (short)  
rechargeable in electric outlet  
for metal charging handle C

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### **Ophthalmoscopes – An in-depth look at the eye**

KaWe ophthalmoscopes are sure to please due to their precise and shadow-free illumination of the fundus of the eye. With unique technology – the wide aperture angle of the illumination optics – the examiner is able to see a large area while at a very close proximity to the subject.

The ophthalmoscope heads of the KaWe EUROLIGHT® and KaWe PICCOLIGHT® product lines have one, five or six apertures. The handles take either KaWe EUROLIGHT® 2.5 V series batteries or rechargeable batteries. Your own individual diagnostic set can be assembled entirely according to your individual preferences.

Our ophthalmoscopes are compliant with EN ISO 10942 standards.

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		KaWe EUROLIGHT®						KaWe PICCOLIGHT®				
		E10	E30	E35	E36	E36	E36	E36	E50	E56	E56	E56
Versorgungsspannung [V]   supply voltage [V]		2,5	2,5	2,5	2,5	2,5	3,5	3,5	2,5	2,5	2,5	2,5
Ladespannung [V]   charge voltage [V]								240 V				
Filter   filter					EU	US	EU	EU			EU	US
Stecker   plug								EU				
Blenden   aperture	mit 1 Festblende   with one fixed aperture	•	•	•					•			
	mit 5 Blenden   with 5 apertures											
	mit 6 Blenden   with 6 apertures				•	•	•	•		•	•	•
Blendenrad aperture selection dial	mit 18 Korrektionslinsen   with 18 correction lenses	•	•		•	•	•	•	•	•	•	•
Dioptrien   dioptres	+ 20 bis -20   + 20 to -20	•	•		•	•	•	•	•	•	•	•
Kopf   head	Kunststoffkopf   plastic head	•	•	•	•	•	•	•	•	•	•	•
Verschluss   closure	Clic-Verschluss   click-on connection		•		•	•	•	•				
	Verschluss mit Schraube   connection with fastening screw	•										
	Schraubverschluss   screw-on connection								•	•	•	•
Griff   handle	Metallgriff   metal handle	•	•	•	•	•	•	•				
	Kunststoffgriff   plastic handle								•	•	•	•
Leuchtmittel illuminant	Xenon/Halogen-Lampe   xenon/halogen bulb				•	•	•	•			•	•
	Vakuum-Lampe   vacuum bulb	•	•						•			
	LED-Lampe   LED bulb			•						•		
Lebensdauer Leuchtmittel service life of illuminant	ca. 15 Std.   approx. 15 h	•	•						•	•		
	ca. 20 Std.   approx. 20 h				•	•	•	•			•	•
Lichtregler light regulator	stufenlos dimmbar   can be dimmed steplessly	•	•		•	•	•	•				
aufladbar in rechargeable in	Ladestation KaWe MedCharge® 4000 (mit KaWe Ladebatterie) KaWe MedCharge® 4000 charging station (with KaWe rechargeable battery)	•	•		•	•	•					
	Steckdose   electric outlet							•				
mögliche Energiequelle energy source options	Batterie(n)   battery (batteries)	•	•		•	•			•	•	•	•
	KaWe-Ladebatterie   KaWe rechargeable battery	•	•		•	•	•	•				
im Lieferumfang enthalten items included	Ladebatterie 3,5 V   rechargeable battery 3.5V						•	•				
	Reißverschlusstasche   bag with zipper	•	•		•	•	•	•				
	Stofftasche   cloth bag								•	•	•	•
	Hartschalen-Box   hard-top box			•								
erhältlich in   available in	night / sky / stone								•	•	•	•

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### Metal handle – dual mode

① with rechargeable battery = **charging handle**  
for KaWe MedCharge® 4000 charging station  
② with batteries = **battery handle**

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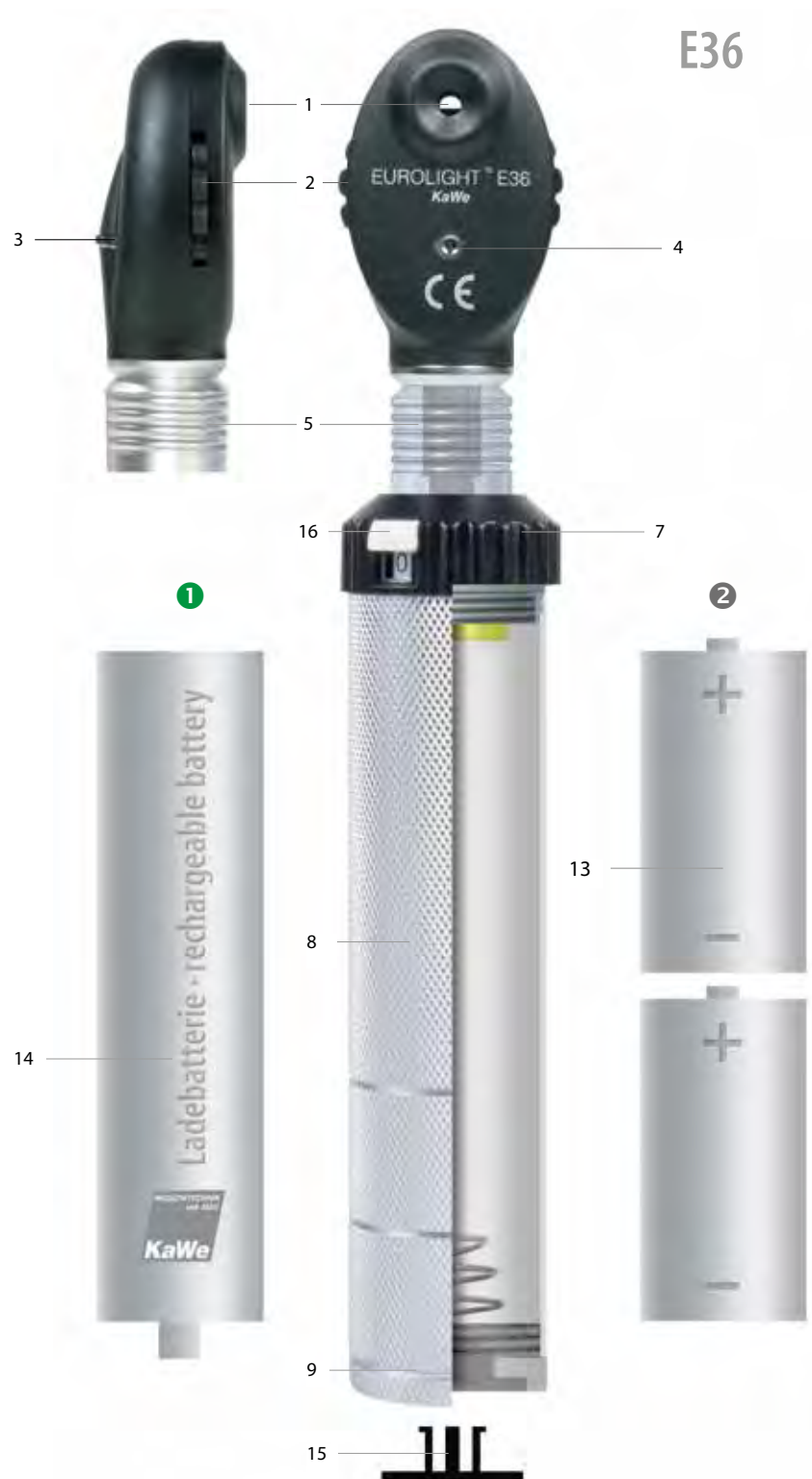
#### Charging station

complete with power  
supply unit and universal  
power plug for  
Lithium-ion technology

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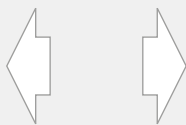
- 1 Observation window
- 2 Lens correction dial
- 3 Aperture selection dial
- 4 Display adjustment
- 5 Click-on connection
- 6 On/Off switch
- 7 Light regulator, can be dimmed steplessly (except for LED)
- 8 Battery/charging handle
- 9 Metal cap
- 10 Positioning/ fastening ring
- 11 Battery handle
- 12 Metal clip
- 13 Batteries
- 14 Rechargeable battery
- 15 End cap
- 16 Push-button switch

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### Function and design at their best.

The KaWe EUROLIGHT® is a high-quality ophthalmoscope available with 1, 5 or 6 apertures. The technically mature lens optics inside of the ophthalmoscope head enable an optimal image quality with no shading. The xenon/halogen, vacuum and LED bulbs ensure authentic colour reproduction.

**Good and uniform illumination! Very large examination field of the fundus of the eye.**

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- click-on connection
- vacuum bulb
- metal battery/charging handle C
- dimmable rheostat
- with lens correction dial  
incremental steps from +/- 20 dioptres
- in bag with zipper
- batteries: 2 C batteries, 1.5V
- 1 aperture

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- connection with fastening screw
- vacuum bulb
- metal battery/charging handle C
- dimmable rheostat
- with lens correction dial  
incremental steps from +20 to -20 dioptres
- in bag with zipper
- batteries: 2 C batteries, 1.5V
- 1 aperture

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- click-on connection
- metal battery handle C
- 2.5 V LED bulb
- colour temperature: 5,000 K
- service life of LED approx. 50,000 h
- colour rendering index  $R_a$  ca. >80
- precision optics
- shadow-free examination of the fundus of the eye
- batteries: 2 C batteries, 1.5 V
- 5 apertures
- in hard-top box

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XH Xenon/Halogen-Lampe / xenon/halogen bulb

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## night. sky. stone. A new focus

The coat pocket ophthalmoscope. Small, easy and yet exact – this is the KaWe PICCOLIGHT®. A high-capacity xenon/halogen, vacuum and LED lamps ensures the excellent illumination of the eye fundus. Its elegant design and the classy colours make the KaWe PICCOLIGHT® a medical instrument that is a pleasure to use day in and day out.

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**Features:**

- direct ophthalmoscopy
- with six apertures for all types of ophthalmoscopic examinations
- aperture selection dial with 18 correction lenses from +20 to -20 dioptres
- aspherical precision optics

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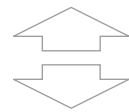
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
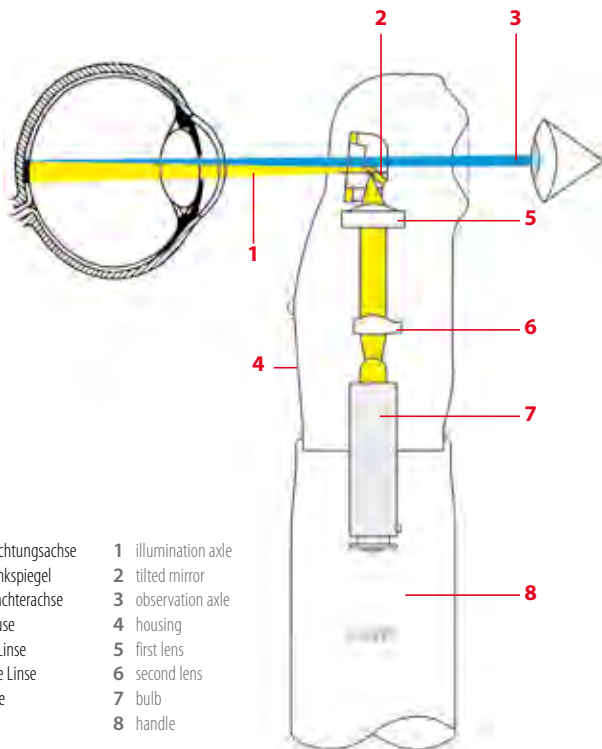
KaWe EUROLIGHT® and KaWe PICCOLIGHT® will please you not only with their compact and appealing design, but also with their sophisticated and highly-developed optical technology inside of their respective lens heads. Thus, in all variants of the 6-aperture ophthalmoscopes, 18 accurately harmonised lenses enable optimal focussing of the light beam. Thereby, the angle of vision of the illumination is  $\pm 7^\circ$  and thus ensuring a large illumination field and an examination area on the retina that is free of shadows.

Regardless of whether you choose a unit with one or with six apertures, the long-lasting xenon/halogen, vacuum or LED incandescent lamp will always ensure an extremely bright light with great colour reproduction.

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Charging station  
only for KaWe EUROLIGHT®  
dermatoscope

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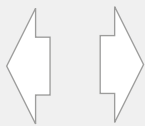
### Features:

- approx. 10-times magnification
- metal handle with click-on connection

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### Features:

- approx. 10-times magnification
- battery handle with screw connection

**Large contact disk (Ø 25 mm) with scale setting for the measurement of pigmentary skin modifications, especially for the early diagnosis of malignant tumours!**

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- click-on connection
- xenon/halogen bulb
- with approx. 10-times magnification
- focus adjustment -6 up to +3.5 dpt
- uniform, intensive illumination
- metal battery/ charging handle C
- dimmable rheostat
- in bag with zipper
- includes 1 bottle of Derma-Gel, 250 ml
- batteries: 2 C batteries ⓧ

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- click-on connection
- **rechargeable in electric outlet (EU 240 V)**
- standard **3.5 V** LED bulb
- illuminance approx. 15,000 Lux
- life of bulb approx. 50,000 h
- includes rechargeable battery **3.5 V**
- with approx. 10-times magnification
- focus adjustment -6 up to +3.5 dpt
- uniform, intensive illumination
- metal charging handle C
- zippered storage bag included
- includes 1 bottle of Derma-Gel, 250 ml

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- vacuum bulb
- with approx. 10-times magnification
- focus adjustment -6 up to +3.5 dpt
- uniform, intensive illumination
- battery handle AA, plastic, black
- in cloth bag
- batteries: 2 Mignon (type AA) ⓧ

3 D | 2,5 V | VL | night

REF 01.33100.021

Leuchtmittel | illuminant

REF 12.75112.003

⚡ auffadbar in Ladestation KaWe MedCharge® 4000 in Verbindung mit Ladebatterie / rechargeable in KaWe MedCharge® 4000 charging station with rechargeable battery  
 ⓧ Batterien sind nicht im Lieferumfang enthalten! / Batteries are not included!  
 ■ Ladebatterie im Lieferumfang enthalten / rechargeable battery is included

XH Xenon/Halogen-Lampe / xenon/halogen bulb  
 VL Vakuum-Lampe / vacuum bulb  
 LED LED-Lampe / LED bulb

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Moistening the skin with the special dermatoscope gel prevents cutaneous reflexes and makes the skin's texture invisible so that only true skin discoloration can be seen. Additional characteristics: hypoallergenic, free of grease, smooths out rough skin very well, excellent colour reproduction, can be wiped off easily, better than oil, water soluble and formaldehyde-free, will not stain skin or clothing.

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Ø 25 mm

**REF 01.99010.001**

## Ladebatterie 3,5 V Rechargeable battery 3.5 V

- kurz
- aufladbar in Steckdose
- für Metall-Ladegriff C
- short
- rechargeable in electric outlet
- for metal charging handle C

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XH Xenon/Halogen-Lampe / xenon/halogen bulb  
VL Vakuum-Lampe / vacuum bulb  
LED LED-Lampe / LED bulb

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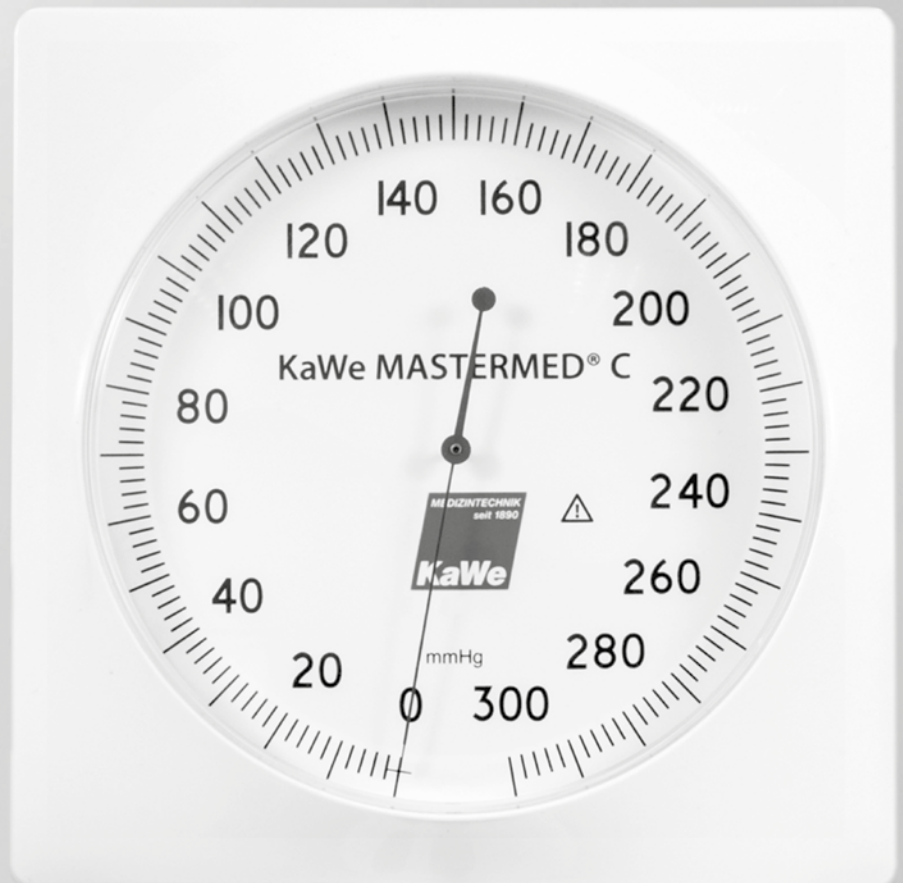
- modern, innovative design
- 24 h operational readiness
- stand-by-mode (memorized light function)
- dimmable touch switch for light control
- longer lifetime of the lamps due to gentle, pulsed start-up phase
- ergonomic handles
- KaWe STS®-Technology with soft-touch handle
- automatic on/off switch
- easy cleaning
- anti-theft system
- easy wall mounting

# KaWe MedCenter® 5000

## WANDSTATION | WALL-MOUNTED DIAGNOSTIC SET

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A combination of high-quality electronics all in one wall-mounted unit – our smart KaWe MedCenter® 5000.

All of the instruments have ergonomic handles with unique dimmable light switches, the user-friendly "soft-touch-system" (KaWe STS®). With this STS technology, soft push buttons are used to adjust the light intensity as needed. The last used light intensity setting is saved even after the handle is removed (memorized light function). The instruments also have a theft-deterrent system to guard against unauthorized use. The KaWe MedCenter® 5000 wall-mounted station combines high technical quality with reliability and user safety.

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Basic and add-on module, 2 heads, 2 handles,  
with exchangeable mains plug  
incl. wall-mounting set

### Otoskop-Kopf\*, night

### Otoscope head\*, night

REF 01.73550.021

KaWe PICCOLIGHT® F.O. LED high power  
LED high power-Lampe  
3-facher Lupenvergrößerung

KaWe PICCOLIGHT® F.O. LED high power  
LED high power bulb  
3X-magnification

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KaWe PICCOLIGHT® E56, EU  
xenon/halogen bulb  
6 apertures  
green filter  
with lens correction dial in steps  
from +20 to -20 dioptres

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KaWe PICCOLIGHT® E56, US  
xenon/halogen bulb  
6 apertures  
blue filter  
with lens correction dial in steps  
from +20 to -20 dioptres

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only to be used with 2.5/4.0 mm ear funnels  
for KaWe PICCOLIGHT® otoscopes  
alternatively, the practical ear funnel module  
REF 02.84001.002 may also be ordered

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**Note:** The KaWe MedCenter® 5000 can be used with a maximum of three handles!

**Wall mounting set** consisting of:  
1 x mounting fixture, 5 screws with anchors, exchangeable mains plug EU (240V), US (110V), GB (240V)

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**Wall-mounted model**  
**Aneroid blood pressure measuring device,** for use with the wall-mounted KaWe MedCenter® 5000 work station, 150 mm scale diameter, scaling: 0-300 mmHg, mounting bracket swivels 80°, screws and wall anchors are included, spiral hose with a range of 3 m further features (►122)

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**XH** Xenon/Halogen-Lampe / xenon/halogen bulb

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KaWe COMBILIGHT® F.O.30 LED  
LED high power bulb  
illuminant REF 12.75253.003  
LED illumination  
with 3X-magnification

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KaWe EUROLIGHT® E36  
xenon/halogen bulb  
6 apertures  
green filter  
illuminant REF 12.75232.003  
with lens correction dial  
incremental steps from +20 to -20 dioptres  
aspherical precision optics

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click-on connection  
dimmable  
charging handle C, medium, incl. Li-ion  
rechargeable battery, only to be used with the  
KaWe MedCharge® 4000 charging station (»189)

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## The great KaWe quality guarantee

Whether during an operation or in an emergency, there is a race against time. In times like these, everything must run smoothly and everyone must know what needs to be done. Above all, anaesthesiologists and doctors need instruments that they can rely on – instruments that are made for critical times like these. KaWe laryngoscopes offer the ideal solution: the most modern technology, the highest-quality standard – be they LED, fibre optic, conventional or single-use models – KaWe offers one of the largest laryngoscope product lines worldwide!

Our new KaWe Med-Charge® 4000 charging station – internationally usable – for 2.5 V or 3.5 V handles rounds out our product line. In addition to our high level of quality, we also offer an excellent cost/effectiveness ratio combined with reliability and optimum safety.

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In the mouth and throat, the lighting conditions are very poor. In order to accommodate for these conditions, KaWe has created a safe and easy intubation method using advanced light technology (LED – with up to 70 % more light) and the latest F.O. light guides. All KaWe fibre optic laryngoscopes have a powerful LED light (2.5 V or 3.5 V) in their handles. The tried and tested xenon light is still available as well. You can, by the way, update the xenon lamp in your handle simply by replacing it with an LED lamp (no other components are required). An extensive selection of F.O. blades from the fibre optic line (Macintosh, Miller, Foregger, KaWe FLAPLIGHT®, TEPRO, KaWe MEGALIGHT®, Economy, Polio) makes it possible to meet the needs of even very difficult anatomic conditions. Macintosh blades are available with integrated as well as with exchangeable fibre optics. Our technicians place a particular importance on easy handling and guarantee that the blades can be exchanged quickly. All blade types are made of stainless steel, are compliant with DIN ISO 7376 standards and the reusable blades can be sterilized in an autoclave at 134°C.

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**Metal handle – dual mode**  
 ① with rechargeable battery = **charging handle**  
 for KaWe MedCharge® 4000 charging station  
 ② with batteries = **battery handle**

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#### Features of the fibre optic bundles:

- high transmission factor within the entire visible wavelength range
- numerical aperture 0.64
- excellent radiation properties
- more than 4000 sterilization cycles in the autoclave at 134°C
- 3500 / 5500 / 8000 single fibres minimum per blade type ensure optimal light emission

#### Features of the xenon lamp:

- xenon lamp with a service life of more than 20 hours
- metal protective lamp cap ensures that the xenon lamp has a longer service life

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#### With xenon illumination

- all handles are compatible according to DIN ISO 7376
- metal light sockets, advantage: xenon-bulb can be removed very quickly to be sterilized
- light-weight (no casing or interior socket)
- all KaWe 2.5 V (small and medium) fiber optic xenon handles perform dual functions
  - with batteries = battery handle
  - with rechargeable battery = charging handle for KaWe MedCharge® 4000 charging station
- very long rechargeable battery life 2.5 V + 3.5 V (NiMH)
- environmentally-friendly
- all KaWe 3.5 V (small + medium) fiber optic xenon handles can be used only with rechargeable battery and charged with the KaWe MedCharge® 4000
- the 3.5 V lithium-ion rechargeable battery has a high energy density

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### With LED illumination

- all handles are compatible according to DIN ISO 7376
- metal light sockets, advantage: LED light can be removed very quickly to be sterilized
- light-weight (no casing or interior socket)
- all KaWe 3.5 V (small + medium) fibre optic LED handles can be used only with rechargeable battery and charged with the KaWe MedCharge® 4000

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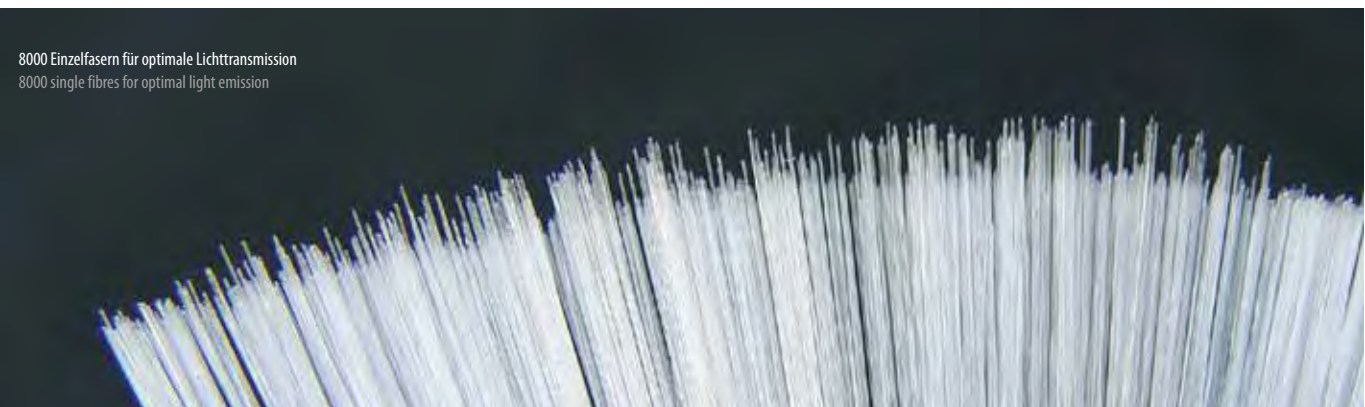
## 3,5V XENON | Ladegriffe | charging handles

6	<b>3,5V</b>	Ladegriff AA, klein (NiMH)	charging handle AA, small (NiMH)	<b>REF 03.41015.711</b>
	∅	19 mm	19 mm	
	XL	Xenon-Lampe	xenon bulb	
		inkl. NiMH Ladebatterie 3,5 V	inkl. 3.5 V NiMH rechargeable battery	
7	<b>3,5V</b>	Ladegriff C, mittel (Li-Ion)	charging handle C, medium (Li-Ion)	<b>REF 03.41010.521</b>
	∅	28 mm	28 mm	
	XL	Xenon-Lampe	xenon bulb	
		inkl. Li-Ion Ladebatterie 3,5 V	inkl. 3.5 V Li-ion rechargeable battery	
		<b>Vorteile Li-Ion-Ladebatterien:</b> - hohe Lebensdauer - hohe Energiedichte - kein Memory-Effekt - geringe Selbstentladung	<b>Advantages of Li-Ion rechargeable batteries:</b> - long life - high energy density - no memory effect - little self discharging	<b>REF 12.80220.722</b>

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# F.O. LED – Batteriegriffe | battery handles

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LED illumination is the optimal prerequisite for easy intubation. The LED illuminants feature excellent light efficiency and superb light transmission. Try them out for yourself!

2,5V LED		Batteriegriffe   battery handles		
1	2,5V ∅ LED ☒	Batteriegriff AA, klein 19 mm LED standard Batterien (Mignon AA)	Battery handle AA, small 19 mm LED standard batteries (AA)	REF 03.41020.711
2	2,5V ∅ LED ☒	Batteriegriff C, mittel 28 mm LED standard Batterien (Baby C)	Battery handle C, medium 28 mm LED standard batteries (C)	REF 03.41020.721
3	2,5V ∅ LED ☒	Batteriegriff AA, kurz 32 mm LED standard Batterien (Mignon AA)	Battery handle AA, short 32 mm LED standard batteries (AA)	REF 03.41020.741
2,5V LED		Einmal-Batteriegriff   Single-use battery handle		
4	2,5V ∅ LED ☒	Batteriegriff C, mittel Kunststoff, VE = 10 St. 30 mm LED standard Batterien (Baby C)	Battery handle C, medium plastic, PU = 10 pcs. 30 mm LED standard batteries (C)	REF 03.41022.721
2,5V LED		Leuchtmittel für KaWe F.O. Griffe   Illuminant for KaWe F.O. handles		
	LED	LED standard, VE = 1 Stück	LED standard, PU = 1 items	REF 12.75154.003

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The 2.5 V LED battery handles are not compatible with the KaWe MedCharge® 4000 charging station. Cannot be used with 2.5 V rechargeable battery.

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81

2,5V LEDhp		Batteriegriffe   battery handles		
5	2,5V Ø LEDhp ⊗	Batteriegriff AA, klein 19 mm LED high power Batterien (Mignon AA)	Battery handle AA, small 19 mm LED high power batteries (AA)	REF 03.41030.711
6	2,5V Ø LEDhp ⊗	Batteriegriff C, mittel 28 mm LED high power Batterien (Baby C)	Battery handle C, medium 28 mm LED high power batteries (C)	REF 03.41030.721
7	2,5V Ø LEDhp ⊗	Batteriegriff AA, kurz 32 mm LED high power Batterien (Mignon AA)	Battery handle AA, short 32 mm LED high power batteries (AA)	REF 03.41030.741

2,5V LED		Leuchtmittel für KaWe F.O. Griffe   Illuminant for KaWe F.O. handles		
	LEDhp	LED high power, VE = 1 Stück	LED high power, PU = 1 items	REF 12.75151.003

3,5V LEDhp		Ladegriffe   charging handles		
8	3,5V Ø LEDhp ⚡	Ladegriff AA, klein (NiMH) 19 mm LED high power inkl. NiMH Ladebatterie 3,5V	charging handle AA, small (NiMH) 19 mm LED high power incl. 3.5 V NiMH rechargeable battery	REF 03.41145.711
9	3,5V Ø LEDhp ⚡	Ladegriff C, mittel (Li-Ion) 28 mm LED high power inkl. Li-Ion Ladebatterie 3,5V	charging handle C, medium (Li-Ion) 28 mm LED high power incl. 3.5 V Li-ion rechargeable battery	REF 03.41140.521

3,5V LED		Leuchtmittel für KaWe F.O. Griffe   Illuminant for KaWe F.O. handles		
	LEDhp	LED high power, VE = 1 Stück	LED high power, PU = 1 items	REF 12.75251.003

# F.O. Spatel | F.O. blades

FÜR F.O. GRIFFE | FOR F.O. HANDLES

82

1 (»84)

## KaWe MEGALIGHT® Macintosh F.O.

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integrated fibre optic illumination

Größen

sizes

Lichtleiter

light guide

0, 1, 2, 3, 4, 5

□ 3 x 7 mm

2 (»85)

## Macintosh F.O.

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replaceable fibre optic illumination

Größen

sizes

Lichtleiter

light guide

0, 1, 2, 3, 4, 5

Ø 4 mm

3 (»86)

## TEPRO Macintosh F.O.

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replaceable fibre optic illumination

Größen

sizes

Lichtleiter

light guide

2, 3, 4

Ø 4 mm

4 (»87)

## KaWe FLAPLIGHT® F.O.

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replaceable fibre optic illumination

Größen

sizes

Lichtleiter

light guide

2, 3, 4

Ø 4 mm

5 (»87)

## KaWe FLAPLIGHT® F.O. MEGALIGHT®

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integrated fibre optic illumination

Größen

sizes

Lichtleiter

light guide

2, 3, 4

□ 3 x 7 mm

6 (»88)

## KaWe MEGALIGHT® Miller F.O.

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integrated fibre optic illumination

Größen

sizes

Lichtleiter

light guide

00, 0, 1, 2, 3, 4

Ø 3 mm

7 (»89)

## Miller F.O.

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replaceable fibre optic illumination

Größen

sizes

Lichtleiter

light guide

00, 0, 1, 2, 3, 4

Ø 4 mm

8 (»90)

## Foregger F.O.

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replaceable fibre optic illumination

Größen

sizes

Lichtleiter

light guide

0, 1, 2, 3, 4

Ø 4 mm

9 (»90)

## Polio Macintosh F.O.

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replaceable fibre optic illumination

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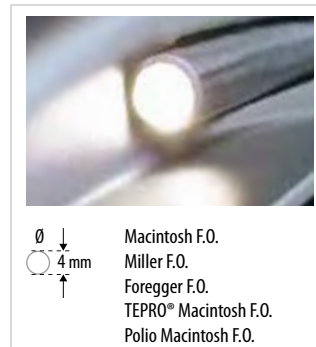
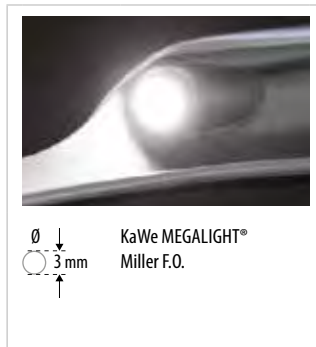
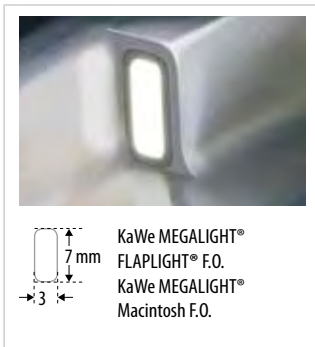
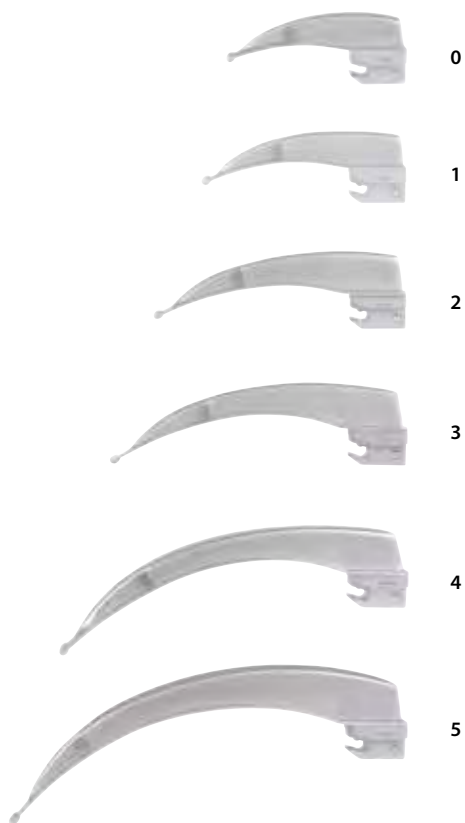
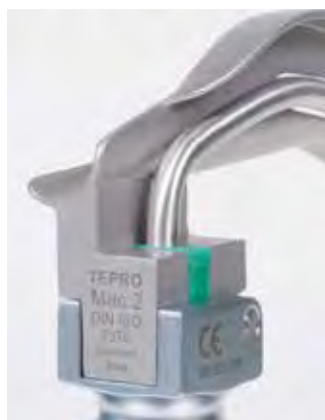
The reliable TEPRO has the same shape as the most widely used Macintosh blades. What is special about them however, is the tapering on the part of the blade near the handle. This recess increases the flexibility and manageability of the blade during intubations.

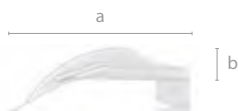
Even with the protection offered the front teeth by the tapering of the proximal blade, the tongue can be pushed up and out of the way as in a normal intubation procedure with a conventional Macintosh blade. The intubation procedure can be carried out with ease and the patient's teeth remain optimally protected.

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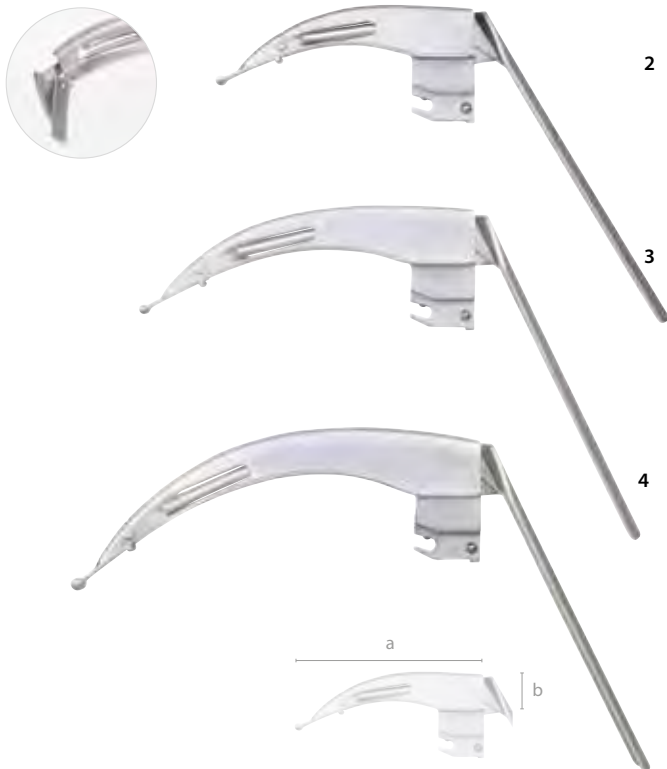
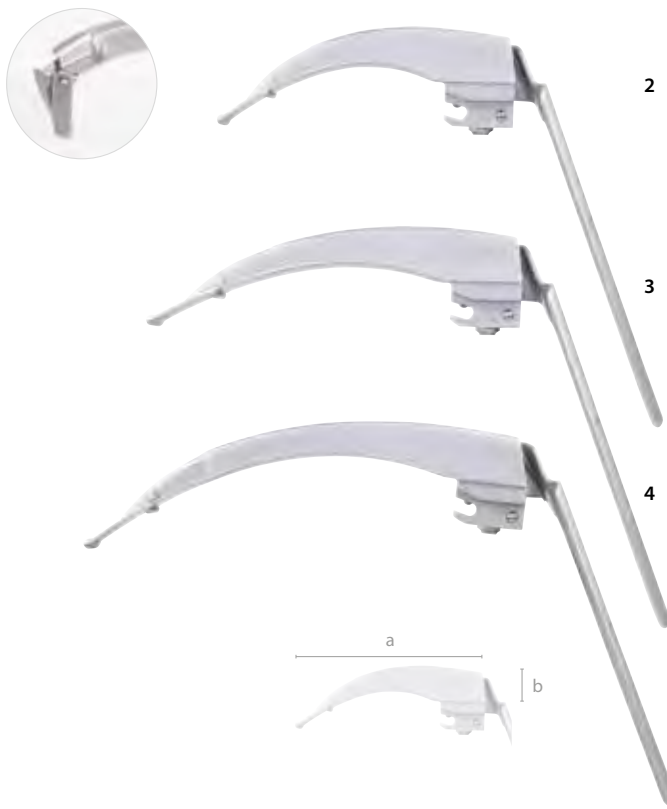
- matte stainless steel – prevents back reflection
- with integrated fibre optics
- no dirt-collecting recesses (easy to clean)
- fibre optics Ø 3.0 mm
- can be treated in the autoclave up to 134°C – approx. 4000 times
- compatible according to DIN ISO 7376
- maintenance-free

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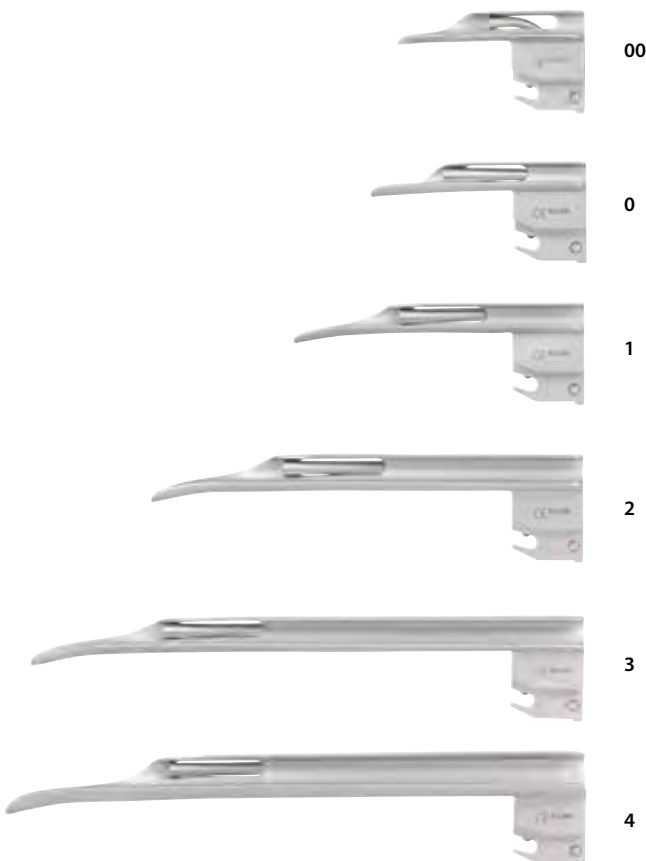
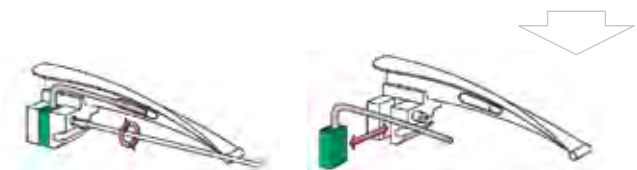
- matte stainless steel – prevents back reflection
- replaceable light guide
- easy to clean
- environmentally-friendly and cost-effective
- fibre optic illumination Ø 4 mm, 5500 single fibres minimum
- can be treated in autoclave up to 134°C – approx. 4000 times
- compatible according to DIN ISO 7376

	Größe   Size	a x b (mm)	Ersatzlichtleiter F.O. Spare light guide F.O.
REF 03.42023.591	00	65 x 11	REF 03.49023.591
REF 03.42023.601	0	75 x 11	REF 03.49023.601
REF 03.42023.611	1	102 x 11	REF 03.49023.611
REF 03.42023.621	2	153 x 13	REF 03.49023.621
REF 03.42023.631	3	195 x 13	REF 03.49023.631
REF 03.42023.641	4	205 x 17	REF 03.49023.641

**Macintosh F.O. / TEPRO® F.O. / Miller F.O. / Foregger F.O. / Polio F.O.**  
Einfaches Wechseln des F.O. Lichtleiters durch Lösen der Schraube  
Fibre-optic light guide easily replaced by loosening the fastening screw

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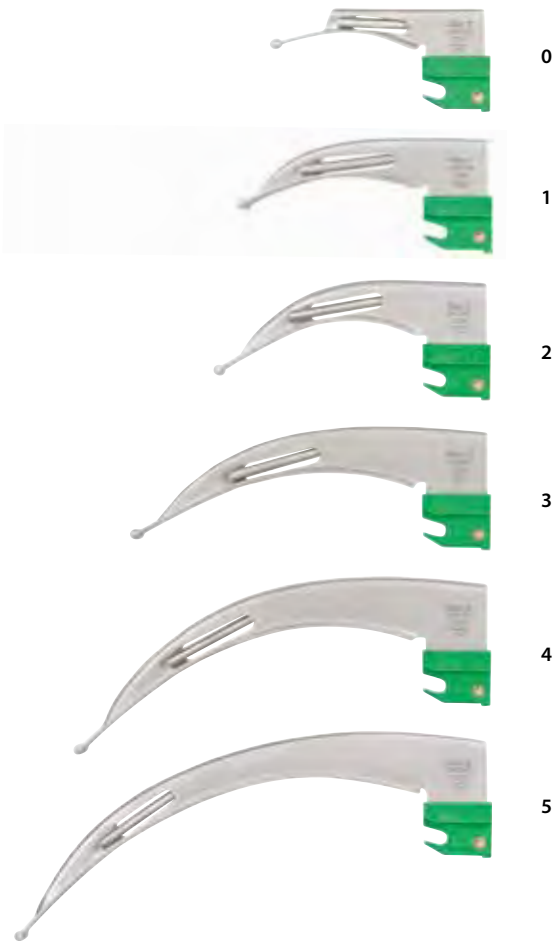
latexfrei  
latex-free

- matte stainless steel – prevents back reflection
- with finely-polished acrylic light guide with excellent light transmission
- light guide Ø 5 mm
- very safe, prevents cross-contamination
- locks firmly into place with two ball fasteners
- light-weight
- compatible according to DIN ISO 7376
- packaged in practical tyvek packaging

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- acrylic light guide (PMMA) ensures optimal light transmission
- brightness >10,800 lux / with 3.5 V
- light guide Ø approx. 5.4 - 5.7 mm
- no spreading of infection
- environmentally-friendly / recyclable plastic (polycarbonate)
- maintenance-free

steril   sterile	unsteril   non-sterile	Größe   Size	a x b (mm)
REF 03.43114.622	REF 03.43014.622	2	114 x 20
REF 03.43114.632	REF 03.43014.632	3	134 x 22
REF 03.43114.642	REF 03.43014.642	4	153 x 22

VE = 10 St. PU = 10 items	VE = 20 St. PU = 20 items
------------------------------	------------------------------

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Lieferung in  
packaged in

REF 03.62022.002

Economy-Batteriegriff  
Economy battery  
handle  
REF 03.41001.721

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REF 03.62720.722

Ladegriff  
Charging handle  
REF 03.41010.521

3,5V | Li-Ion  
Ladebatterie  
Rechargeable battery

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## Proven KaWe quality. Used daily worldwide.

Conventional light laryngoscopes have been a part of the reputed KaWe assortment for many years. This high-performance quality instrument has proven itself in its every day use as an extremely reliable and durable instrument. Our conventional light handles can be used with our conventional light blades (Macintosh, Miller, Foregger, Polio). As with the F.O. models, these instruments are easy to use and the blades are easy to change. We also offer a KaWe FLAPLIGHT® C model in our line of conventional light laryngoscopes, which we have expanded to include single-use blades made of metal. These come as either Macintosh or Miller models and have LED lamps. All blade types are made of stainless steel, are compliant with DIN ISO 7376 standards and the reusable blades can be sterilized as described in the user's manual.

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### Conventional handles:

- all handles are compatible in accordance with DIN ISO 7376
- lightweight (no inner sleeve)
- the KaWe warm light handles (small + medium) have two operating modes:
  - with batteries = battery handle
  - with rechargeable battery = charging handle for charging station KaWe MedCharge® 4000
- rechargeable battery (NiMH) has a very long service life
- environmentally friendly

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**Charging station**  
complete with power  
supply unit and universal  
power plug

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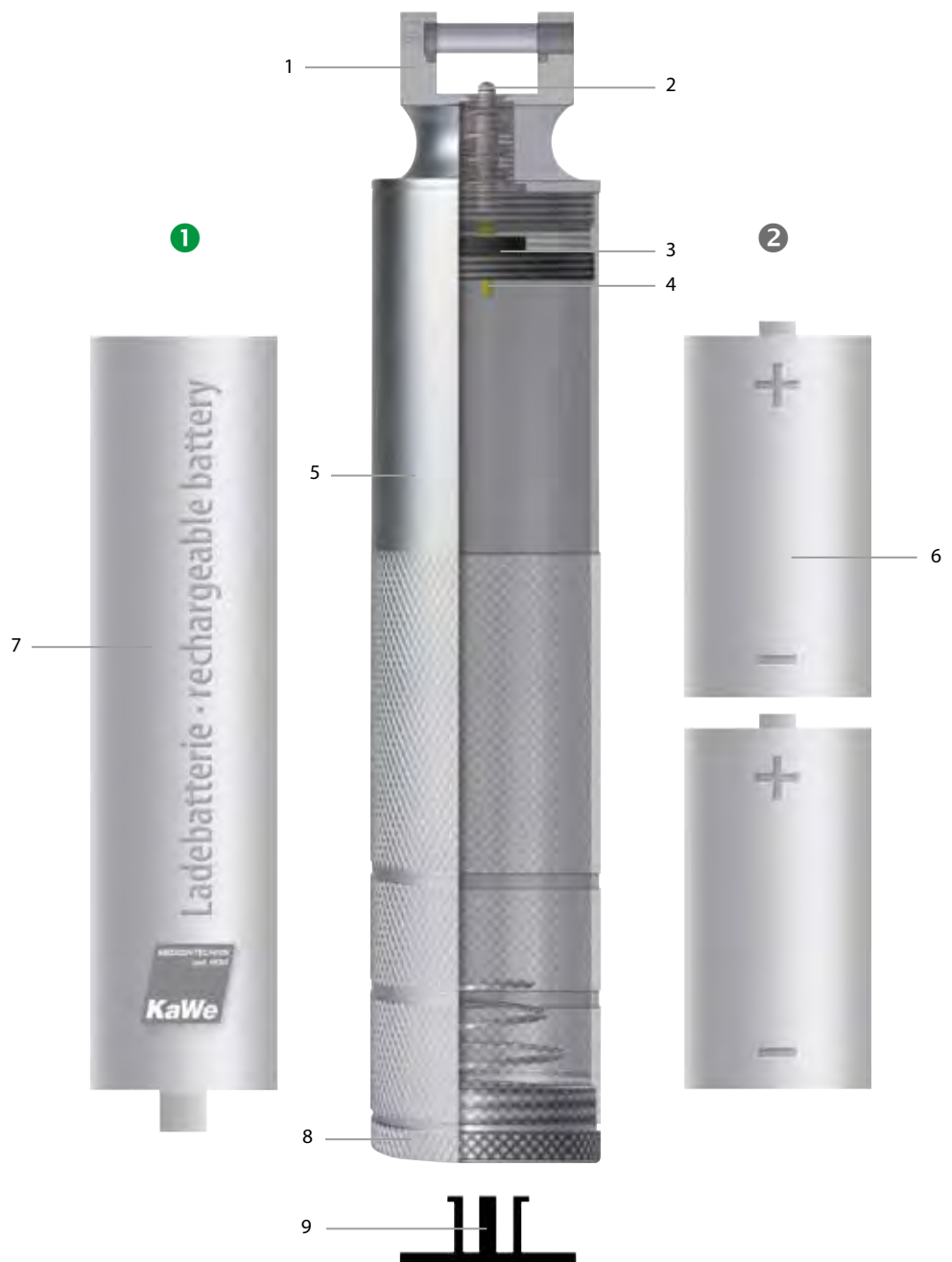
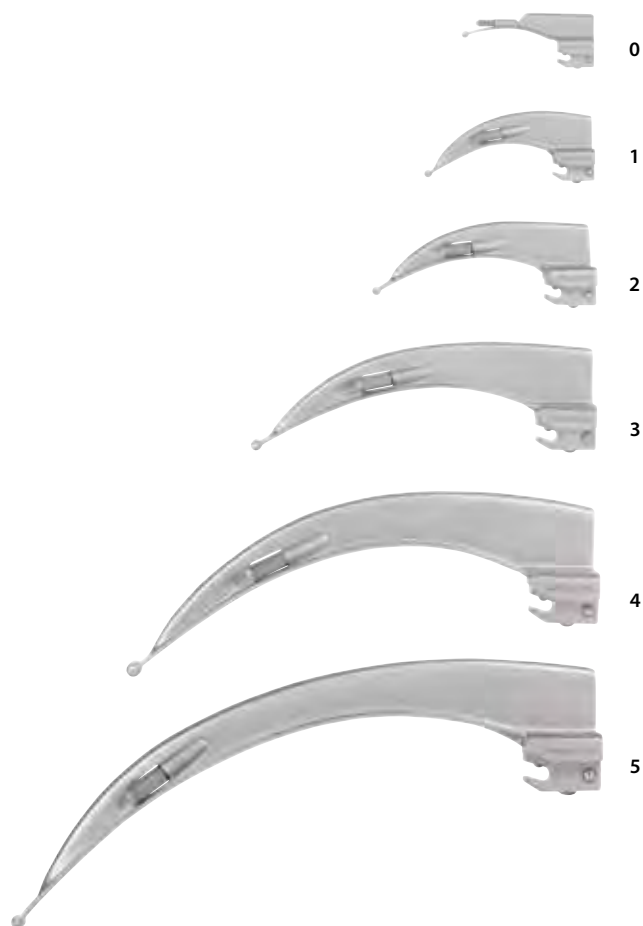
- stainless steel, matte finish, prevents reflection
- vacuum bulb
- brightness  
>1500 and/or  
>2000 lux/2.5 V  
depending on lamp/blade size
- can be sterilized as instructed in the user's manual
- compatible according to DIN ISO 7376

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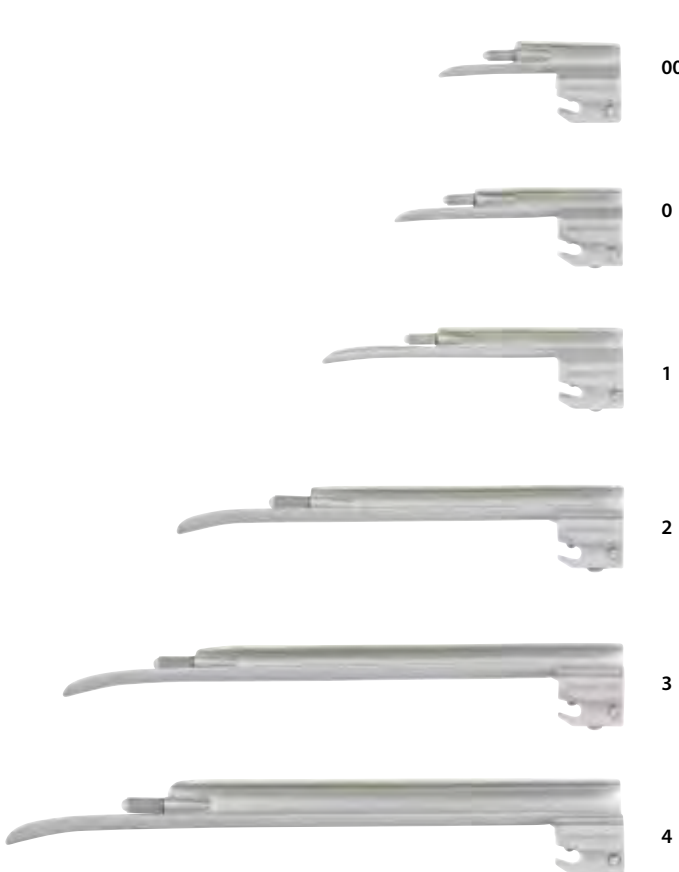
- stainless steel, matte finish, prevents back reflection
- vacuum bulb
- brightness  
>1500 and/or  
>2000 lux/2.5 V  
depending on lamp/blade size
- can be sterilized as instructed in the user's manual
- compatible according to DIN ISO 7376

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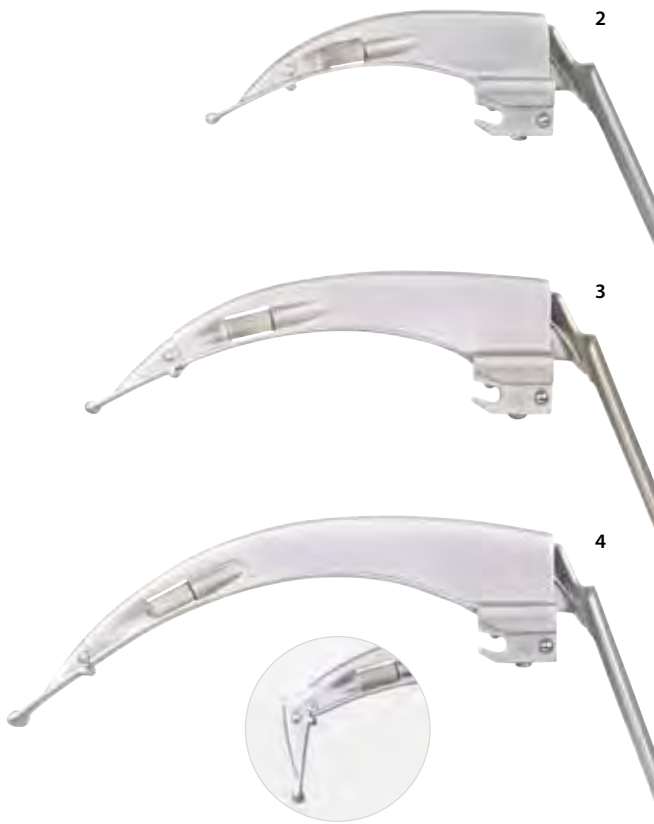
The flexible and up to 60° bendable blade tip makes it easier to raise the epiglottis and makes it possible to insert the instrument with a good line of sight even under the most difficult anatomical conditions. Better vision and less leverage effect reduce the risk of injury to the larynx and teeth. The bulbs can be exchanged effortlessly and individually.

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Improved, easy use on obese patients. For all other features, see the Macintosh C

	Größe   Size	a x b (mm)
REF 03.12060.632	3	130 x 25
REF 03.12060.642	4	150 x 25

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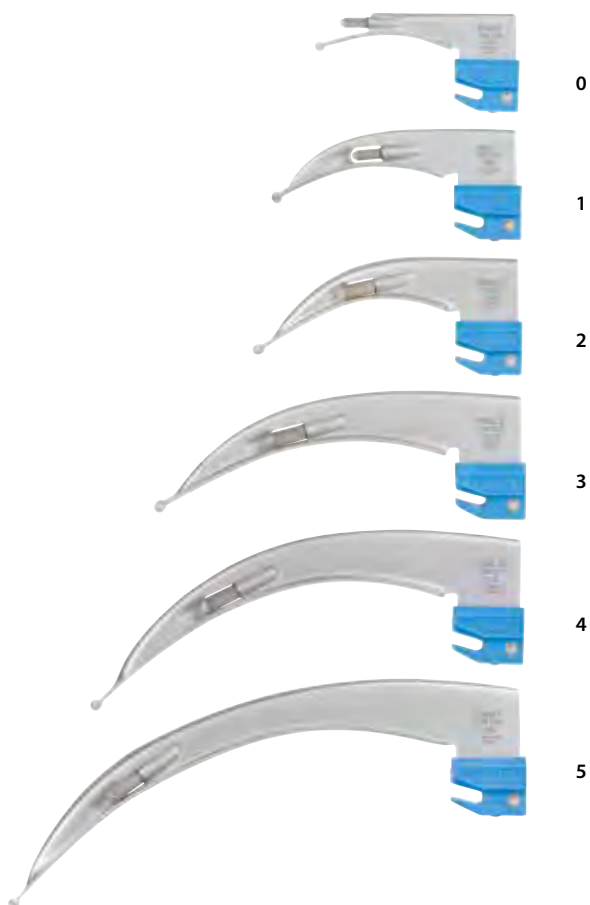
- stainless steel, matte finish with non-breakable plastic socket, same product lifetime as metal
- vacuum bulb
- brightness  
>1500 and/or  
>2000 lux/2.5 V  
depending on lamp/blade size
- environmentally-friendly, economical and inexpensive
- can be sterilized as instructed in the user's manual
- compatible according to DIN ISO 7376

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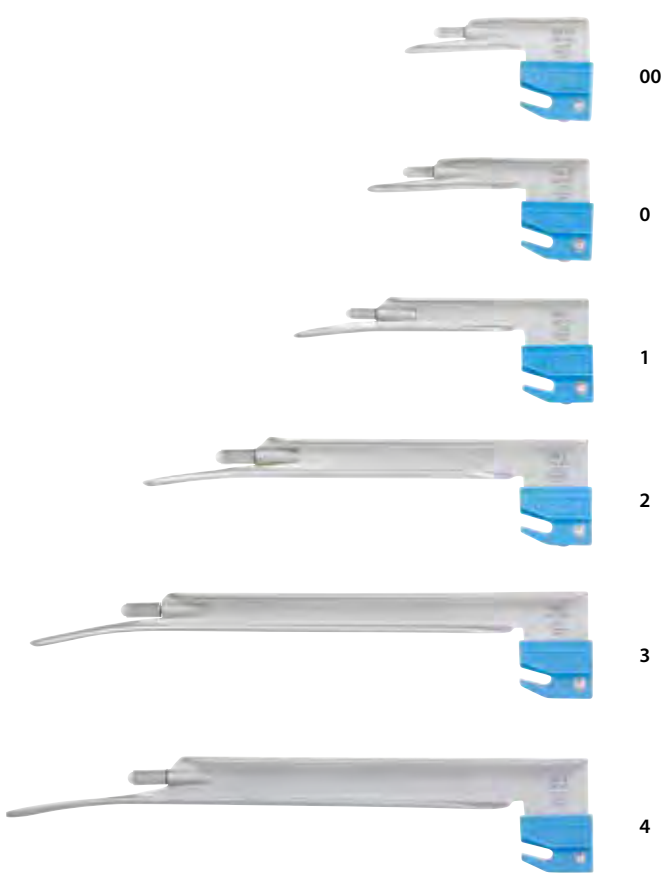
- stainless steel, matte finish with non-breakable plastic socket, same product lifetime as metal
- vacuum bulb
- brightness >1500 and/or >2000 lux/2.5 V depending on lamp/blade size
- environmentally-friendly, economical and inexpensive
- can be sterilized as instructed in the user's manual
- compatible according to DIN ISO 7376

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- stainless steel, matte finish – prevents reflection
- suitable for use in emergency medicine
- powerful LED with white, uniform illumination
- illumination at the blade tip:  
> approx. 6,000 Lux / with 2.5 V
- light guide Ø 5 mm
- very safe, cross-contamination is prevented
- held securely in place with two ball anchors
- lightweight
- compatible according to DIN ISO 7376
- cannot be used with the 2.5 V rechargeable battery
- in practical tyvek packaging

	Größe   Size	a x b (mm)
REF 03.14010.602	0	82 x 15
REF 03.14010.612	1	94 x 21
REF 03.14010.622	2	103 x 21
REF 03.14010.632	3	134 x 22
REF 03.14010.642	4	156 x 24

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- stainless steel, matte finish – prevents back reflection
- suitable for use in emergency medicine
- powerful LED with white, uniform illumination
- illumination at the blade tip: > approx. 6,000 Lux / with 2.5 V
- light guide Ø 5 mm
- very safe, cross-contamination is prevented
- held securely in place with two ball anchors
- lightweight
- compatible according to DIN ISO 7376
- cannot be used with the 2.5 V rechargeable battery
- in practical tyvek packaging

113

	Größe   Size	a x b (mm)
REF 03.14020.592	00	68 x 14
REF 03.14020.602	0	81 x 14
REF 03.14020.612	1	108 x 14
REF 03.14020.622	2	160 x 16
REF 03.14020.632	3	200 x 18
REF 03.14020.642	4	213 x 20

VE = 10 St. | PU = 10 items

# C Laryngoskop-Set | C Laryngoscope set

ERWACHSENE | ADULTS

1 Griff, 3 Spatel | 1 handle, 3 blades

Griff   handle	Spatel   blades
	<b>Economy Macintosh C</b>
C mittel   medium 	Größe   size 2  3  4 
(»104)	(»110)

2,5 V

Lieferung in  
packaged in

REF 03.51022.002

Economy-Batteriegriff  
Economy battery handle  
REF 03.11001.721

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for Macintosh, size: 0, 1, 2, 3, 4  
Miller, size: 00, 0, 1  
Foregger, size: 0, 1

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for Macintosh, size: 2, 3, 4  
Miller, size: 0, 1, 2, 3  
Foregger, size: 0, 1, 2, 3

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for 2 KaWe FLAPLIGHT® blades  
and 1 handle,  
for KaWe FLAPLIGHT®, size: 3, 4

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### Manometer

- Made in Germany
- shock protected in accordance with DIN EN ISO 81060-1
- protection against over-inflation
- scale Ø 60 mm

All KaWe blood pressure measuring devices meet EC Standards 93/42/EEC (standards for medical products) and the product-specific standards DIN EN ISO 81060-1. KaWe blood pressure measuring devices are authorised and certified by our notified body. Thanks to our years of experience and the continuous improvement of our products, we are always in a position to provide our customers with absolutely reliable and precise measuring instruments.

KIRCHNER & WILHELM GmbH + Co. KG is an institution registered with the DAM (German Academy for Metrology). This registration entitles us to carry out metrological tests in accordance with § 11 of the Medical Device Operator Ordinance with regard to non-invasive blood measuring devices (NIB) of all types.

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### Blood pressure measuring: still called “RR” today – in honour of Riva-Rocci

In 1896, the Italian physician Scipione Riva Rocci (1863-1937) demonstrated his method for determining the blood pressure, which is still used today - he had created the prototype of the modern blood pressure measuring device. In his honour we still use the letters “RR” (Riva Rocci), if we are speaking of blood pressure. The term “mmHg” comes from the time when measurements were taken only with a mercury column. Riva Rocci’s method however still only involved feeling for the pulse with the fingers. In 1905, the Russian army surgeon Nikolai Sergejewitsch Korotkow (1874-1920) improved upon this method through use of a stethoscope.

KaWe offers blood pressure measuring devices that you can trust. Our blood pressure measuring devices are user-friendly and set themselves apart from the rest with their superior quality and great value. All KaWe MASTERMED® blood pressure measuring devices are latex-free. Our product line extends from the aneroid blood pressure measuring device with a 1 or 2 hose system, to the KaWe MASTERMED® C Premium blood pressure measuring device, which is available in wall-mounted, table-mounted or instrument stand models. This high-quality blood pressure measuring device can also be used in combination with the KaWe MedCenter® 5000, offering you yet another option for your wall-mounted diagnostic tool set.

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- 1 Protection ring
- 2 Protection glass
- 3 Needle
- 4 Dial instrument with scale setting in mmHg
- 5 Serial number for individual marking

- 6 Air release valve knob
- 7 Palm rest
- 8 Inflation bulb
- 9 Hose
- 10 One piece cuff without bladder
- 11 Cloth bag

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- one-piece cuffs without bladder
- biocompatible in accordance with DIN EN ISO 10993
- with 70 D Nylon fabric, easy cleaning
- extremely resilient: up to 50,000 uses

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The KaWe MASTERMED® C Premium blood pressure monitors provide clinicians the highest level of quality and value. The instruments in this series are defined by their innovative and practical features, smooth surfaces for easy cleaning, and particularly precise gauges. These blood pressure monitoring devices are available as wall-mounted, table top and freestanding models. The double-walled construction of these devices makes them robust and therefore ideal for use in medical facilities. Each model comes equipped with a cuff holder.

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#### Features:

- complies with DIN EN ISO 81060-1 standards
- large, Ø 150 mm gauge with thin, red dial for exact measurement readings
- numbered instrument dial reduces the risk of parallax
- scaling: 0 - 300 mmHg
- one-piece cuff without rubber bladder
- plug-in adapter for quicker cuff exchange
- 2-year guarantee on device when used properly

Präzisions-Messwerk mit Beryllium-Kupfer-Membran  
Precision measuring system with beryllium-copper diaphragm

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### Aneroid blood pressure measuring device table top model

- integrated handle and bases that prevent scratches
- spiral hose with a range of 3 m

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### Aneroid blood pressure measuring device wall-mounted model

- 150 mm scale diameter
- scaling: 0-300 mmHg
- mounting bracket swivels 80°
- for use with the wall-mounted KaWe MedCenter® 5000 work station
- screws and wall anchors included
- spiral hose with a range of 3 m

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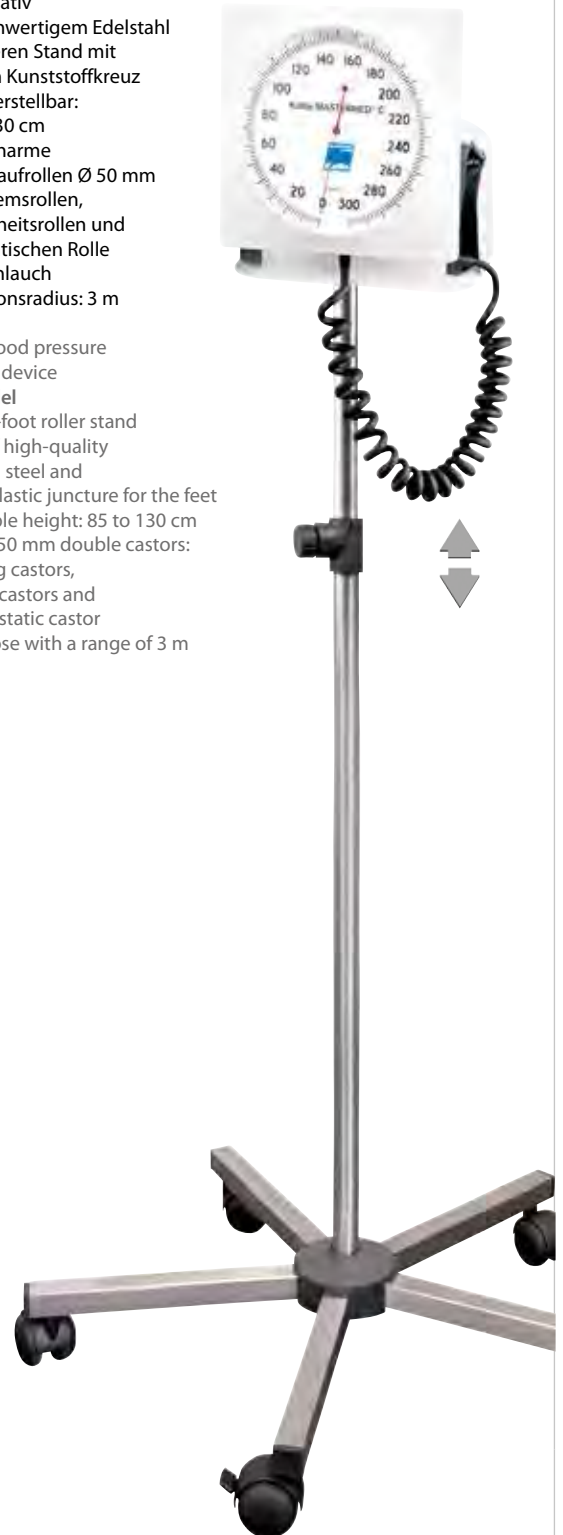
### Aneroid blood pressure measuring device stand model

- stable 5-foot roller stand made of high-quality stainless steel and sturdy plastic juncture for the feet
- adjustable height: 85 to 130 cm
- quiet Ø 50 mm double castors: 2 locking castors, 2 safety castors and one antistatic castor
- spiral hose with a range of 3 m

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SwiSto3 continuous and pulsed current therapy with one machine for the treatment of hand, foot and underarm perspiration. KaWe's 3rd generation, the latest technology and manufacturing experience for more than 20 years.

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The elevated tendency to perspire on the feet, hands and underarms is often very uncomfortable for those affected. Iontophoresis has proven to be an effective, cost-efficient in the long-term, safe and easy-to-use form of therapy for this situation. **Compared to injections or operative therapy, iontophoresis therapy is pain-free. Medications can cause side effects.**

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The new SwiSto3, with its modern design is extremely user-friendly and provides you with two possible treatment options for the treatment of hyperhidrosis:

- therapy with continuous current or
- therapy with pulsed current

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The new SwiSto3 does not have language software. The user is simply guided through the menu and confirms the desired setting by pressing the SET button. The graphic LCD module illuminates shows all of the setting in a clear and organized manner. The user can change the current intensity unaided during the treatment, so that neither a second person nor a foot switch are necessary. The automatic current control function prevents the user of this equipment from feeling an electric shock (electric fence effect) when removing their hands and feet from the water trays during treatment. The SwiSto3 is equipped with an automatic shutoff. If the device is not switched off after a treatment is finished, it switches itself off automatically after 2 minutes.

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### Prescription of home therapy devices

Iontophoresis therapy must always be started by one's attending physician. After the treatment has begun to produce results, maintenance treatment that is to be performed at home may be recommended. This home therapy must however be regularly monitored by the attending physician (for example, every 3 months). A justified request for insurance coverage of this procedure is to be obtained from the attending physician and submitted to your insurance provider. Depending on the situation, coverage of the cost is very often approved. Should you have questions or problems, we would be glad to assist you.

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Please consult your physician if you have any of the following conditions: cardiac arrhythmia, pregnancy, pacemakers or metal implants, metal-containing intrauterine devices (contraceptive coil), major skin defects that cannot be covered with Vaseline, insensitivity to painful stimuli.

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The product video on our home page is to be used to learn how to use the SwiSto3.

Please be sure to watch this video before using this device for the first time.

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## Use

Iontophoresis treatment for hyperhidrosis manum et pedum is a physical application. It was first described by Boumann and Grunewald-Lentzer in 1952. Continuous or pulsed current cause ion transport through the skin.

The plastic treatment trays are filled with approx. 3-4 cm of luke-warm water, such that the surfaces of the hands and feet are covered. Before filling the trays, the electrodes must be covered with the foam pads in order to prevent direct contact with the skin. The sponge pockets used for underarm treatment are to be soaked thoroughly with water before use. The treatment should be performed at least 3 times per week with an average treatment time of 15 to 20 minutes until the desired healing results have been achieved. Usually, this takes place after approx. 10-15 treatments. Thereafter, maintenance treatments should be performed regularly (at least once a week) in order to maintain the successful therapy results. The current that flows through the electrode plates can, depending on application (hands, feet, underarms), be adjusted up to a maximum of 25 mA. We recommend, that the polarity be switched if the skin is thickly calloused as the anode is said to have a stronger empirical efficacy.

## Additional healing results

In addition, scientific test have shown that this therapy can also lead to excellent treatment results for gram-negative foot infection, dyshidrotic eczema, pustulosis palmaris et plantaris, acrocyanosis and osteoarthritis.

Please consult your physician if you have any of the following conditions:

- Cardiac arrhythmia
- Pregnancy
- Pacemakers or metal implants
- Metal-containing intrauterine devices (contraceptive coil)
- Major skin defects that cannot be covered with Vaseline
- Insensitivity to painful stimuli

## Cleaning and disinfection

The plastic water trays and the electrodes are to be cleaned thoroughly after each use with disinfecting agent. For hygienic reasons, the foam pads and sponge pockets are only to be used by one and the same patient. If possible, the use of skin crèmes and oils should be avoided during treatments.

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- for the treatment of underarm perspiration
- made of stainless-steel
- incl. sponge pockets, biocompatible
- incl. cable

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- biocompatible
- for plate electrodes
- for the treatment of underarm perspiration

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Auscultation is a vitally important examination procedure that enables a swift diagnosis of a patient's condition. Auscultation is the perception of sound phenomena that are generated from various body organs. This is provided by stethoscopes, which are a trademark tool for any medical personnel.

Auscultation, provided at the thorax, discloses a vast amount of diagnostic information regarding the cardiac and pulmonary function. The auscultation of specific arteries can reveal information about possible stenotic processes. The functional gastric and bowel status may be examined by auscultation of the abdomen and certain orthopaedic joint disorders can be tracked by an auscultation during limb movements.

As the sound pressure levels at the body's surface are usually low, stethoscopes act as highly sensitive acoustic amplifiers. Their acoustic design defines the audible sound composition and loudness at the ear of the examiner. As, particularly in cardiologic auscultation, low frequencies (smaller than 50 [Hz]) need to be captured, membrane-free bell-type chest-pieces are applied in the low frequency region. They need to be applied tightly on the body's surface as air leakage would lead to a significant drop of diagnostic volume. In order to measure higher frequencies (up to 1 [kHz]), which is required for example, in pulmonary auscultation, and in order to avoid possible air leakage, membrane chest-pieces are used.

KIRCHNER & WILHELM has extensive experience in the design and manufacture of highly-sophisticated stethoscopes. This company has been designing innovative stethoscopes for over 60 years, thus contributing greatly to medical progress in clinical auscultation.

For this reason, the company has a separate Research and Development Department, in order to insure that technological advancements are made in the field of stethoscope technology.

A large number of improved stethoscope characteristics have emerged out of this company's many years of technological experience:

These include, the design of bell-type chest pieces, which greatly minimize the probability of the occurrence of air leakage, as well as membrane chest pieces, which, due to their oscillation characteristics, enable highly-sensitive acoustic sound transmission.

The bell and membrane combination stethoscopes can be switched from one sound sensor function to the other without compromising the acoustic characteristics of the stethoscope. A broad selection of sound conducting tubing as well as ear pieces that can be individually adjusted, ensure unaltered sound transmission to the ears of the examiner.

KaWe offers a broad range of ear tip designs, making it possible for the examiner to find an exact but comfortable fit, while at the same time maintaining the required acoustical integrity. This becomes particularly important with an increased number of patient examinations.

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The Planet stethoscope has been our most successful cardiology stethoscope for more than 35 years. Each one is unique and is meticulously fabricated in our factory. The **individual serial number** stamped into the gold-coloured chest piece of each instrument guarantees flawless traceability. Our Planet stethoscope has many impressive features:

- The solidity of the chest piece and substantial weight of 145 gr. all in a chest piece that is only 23mm high (flat construction) eliminates sounds from the fingers and produces an excellent resonance; no ambient noise or sound from the instrument itself.
- Non-locking and air-tight turning mechanism, two different frequency ranges.
- Oscillating high-performance diaphragm (Ø 42 mm) for even the softest auscultation noises.
- Flexible, exchangeable bell for original sound, anti-chill.
- A dual-tube system that contributes to the excellent sound quality.

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- dual-head stethoscope with a flexible, adjustable head piece
- with an oscillating high-performance diaphragm Ø 42 mm
- total length approx. 75 cm
- reversible
- superior sound quality
- with rotatable ear tips

Gewicht ca. | weight approx. 282 gr.

REF 06.22900.022

### Planet-Bruststück Planet chest piece

Ø 42 mm

REF 06.95290.001

### Hochleistungsmembrane High-performance diaphragm

Ø 42 mm

REF 06.96290.001

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Schlauchlänge ca. 60 cm  
tube length approx. 60 cm

REF 06.93265.021

### Ohroliven Ear tips

drehbar  
rotatable

Ø 5,5 mm

REF 06.91111.021

# Prestige

AUS EDELSTAHL | MADE OF STAINLESS STEEL

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For many years, this has been KaWe's classic among its stethoscopes. The precision chest piece is made of solid stainless steel. The tubes, which are available in many different colours, with their integrated stainless steel head pieces offer the highest level of wearer comfort thanks to their soft ear tips. Each stethoscope comes equipped with two spare pairs of ear tips, a spare diaphragm and a name plate.

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## LIEFERUMFANG INCLUDED IN DELIVERY

- Doppelkopf-Stethoskop
- 2 paar Ohroliven, schraubbar
- 1 Ersatzmembran
- Gewicht Standard: ca. 183 gr.
- Gewicht Child: ca. 137 gr.
- Gewicht Baby: ca. 128 gr.
- dual-head stethoscope
- 2 pair of ear tips, screw-on
- 1 spare diaphragm
- weight Standard: approx. 183 gr.
- weight Child: approx. 137 gr.
- weight Baby: approx. 128 gr.

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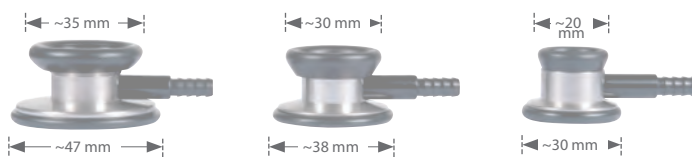
- Standard-Prestige stethoscope**
- stainless steel
  - reversible
  - total length approx. 78 cm

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- Prestige chest piece**
- diaphragm and bell rings made of high-quality biocompatible medical grade silicone

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- Ohrbügel mit Schlauch  
Head piece with tube**
- REF 06.94270.011
  - REF 06.94270.021
  - REF 06.94270.031
  - REF 06.94270.041
  - REF 06.94270.091
  - REF 06.94270.171
  - REF 06.94270.261

Standard-Prestige	
Ø 44 mm	
REF 06.22700.012	
REF 06.22700.022	
REF 06.22700.032	
REF 06.22700.042	
REF 06.22700.092	
REF 06.22700.172	
REF 06.22700.262	

Child-Prestige	
Ø 35 mm	
REF 06.22702.012	
REF 06.22702.022	
REF 06.22702.032	
REF 06.22702.042	
REF 06.22702.092	
REF 06.22702.172	
REF 06.22702.262	

Baby-Prestige	
Ø 28,5 mm	
REF 06.22701.012	
REF 06.22701.022	
REF 06.22701.032	
REF 06.22701.042	
REF 06.22701.092	
REF 06.22701.172	
REF 06.22701.262	

- rot | red
- schwarz | black
- blau | blue
- grün | green **neu | new**
- lila | violet **neu | new**
- burgund | burgundy
- grau | grey **neu | new**

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We have expanded our product line to include aluminium models of our Baby, Child, and Standard-Prestige stethoscopes. Outstanding characteristics such as great sound quality thanks to glass fibre-reinforced, high-performance diaphragms, and the very light weight of the stethoscope are sure to impress anyone. Each stethoscope comes equipped with a pair of replacement ear tips and replacement diaphragm.

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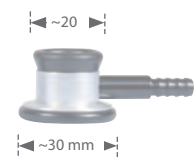
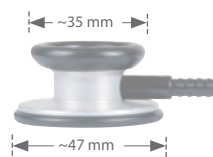
- Prestige light chest piece**
- diaphragm and bell rings made of high-quality biocompatible medical grade silicone

rot   red	
schwarz   black	
blau   blue	
grün   green <b>neu   new</b>	
lila   violet <b>neu   new</b>	
grau   grey	

Standard-Prestige light	
Ø 44 mm	
	REF 06.22710.012
	REF 06.22710.022
	REF 06.22710.042
	REF 06.22710.092
	REF 06.22710.232
	REF 06.22710.262

Child-Prestige light	
Ø 35 mm	
	REF 06.22712.012
	REF 06.22712.022
	REF 06.22712.232
	REF 06.22712.042
	REF 06.22712.092
	REF 06.22712.262

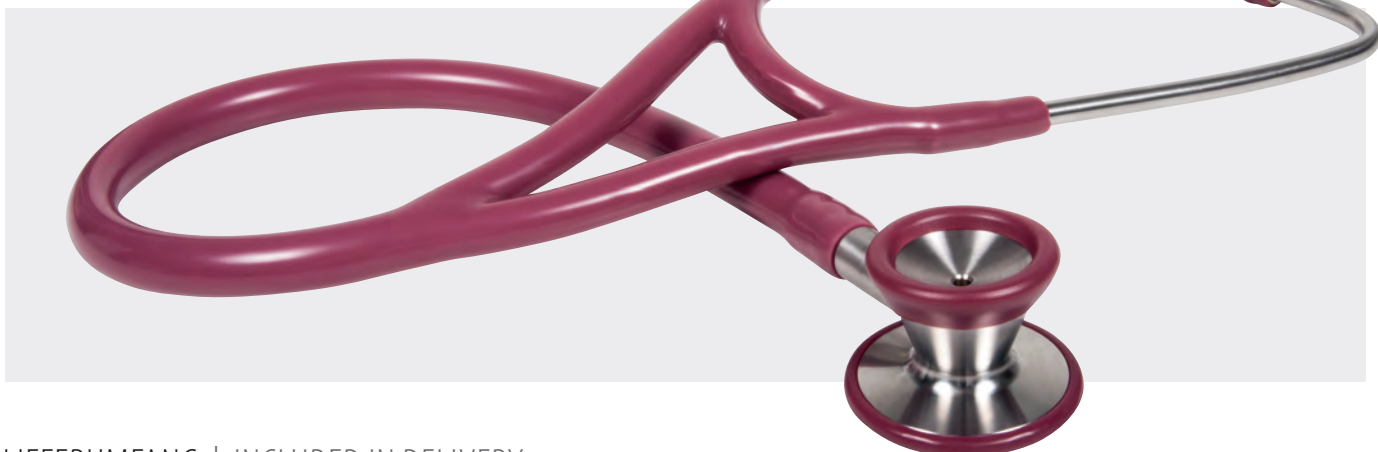
Baby-Prestige light	
Ø 28,5 mm	
	REF 06.22711.012
	REF 06.22711.022
	REF 06.22711.232
	REF 06.22711.042
	REF 06.22711.092
	REF 06.22711.262

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The Profi Cardiology stethoscope is the ideal stethoscope for cardiologists. The chest piece of the KaWe Profi Cardiology stethoscope is available in solid stainless steel. These superior stethoscopes and their outstanding acoustics are crafted with high-quality workmanship and technology. This high-quality stethoscope has a single-tube system with an integrated head piece made of stainless steel. Soft ear tips ensure optimal wearer comfort. Each stethoscope comes with two spare pairs of ear tips, one spare diaphragm and a name plate.

- Profi-Kardiologie Stethoskop
  - Bruststück in massiver Edelstahlausführung
  - Membran Ø 45 mm
  - Gesamtlänge ca. 64 cm
  - umschaltbar
  - Monoschlauchsystem mit integriertem Ohrbügel aus Edelstahl
  - Gewicht: ca. 219 gr.
- Profi Cardiology stethoscope
  - chest piece in solid stainless steel design
  - diaphragm Ø 45 mm
  - total length approx. 64 cm
  - reversible
  - single-tube system with integrated head piece of stainless steel
  - weight: approx. 219 gr.

## LIEFERUMFANG | INCLUDED IN DELIVERY

- Profi-Kardiologie Stethoskop
  - 2 Ohröhlen, schraubbar
  - 2 Ohröhlen, steckbar
  - 1 Ersatzmembrane
  - 1 Namensschild
- Profi Cardiology stethoscope
  - 2 ear tips, screw-on
  - 2 ear tips, plug-in
  - 1 spare diaphragm
  - 1 name plate

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Kirchner COLORSCOP® Duo stethoscopes bring colour into your everyday routine. We use high-quality materials for the production of these stethoscopes. Due to its first-rate workmanship, this stethoscope is suited extremely well for daily use in clinics or by nurses for all types of examinations. The double-sided chest piece can be switched from diaphragm to bell mode by twisting the tube. The exchangeable diaphragm is reinforced with fibre-glass and is therefore especially sensitive. The sound bell is made of metal and therefore guarantees a good resonance because of optimal sound conduction. The available selection of eight different colours almost completely eliminates the possibility of a user mistaking their instrument for that of another.

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- the colourful dual-head stethoscope, reversible, lightweight
- diaphragm Ø 43.8 mm
- total length approx. ca. 78 cm
- weight: approx. 110 gr.

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The Double dual-head stethoscope is the less expensive alternative to the Kirchner COLORSCOPE® Duo. The chest piece is made of aluminium and is therefore especially lightweight.

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The Rapport dual-head stethoscope with its excellent acoustics has been proving its worth for years. This stethoscope is extremely popular among students because it is available in many different colours. Each stethoscope comes with an extensive set of spare parts.

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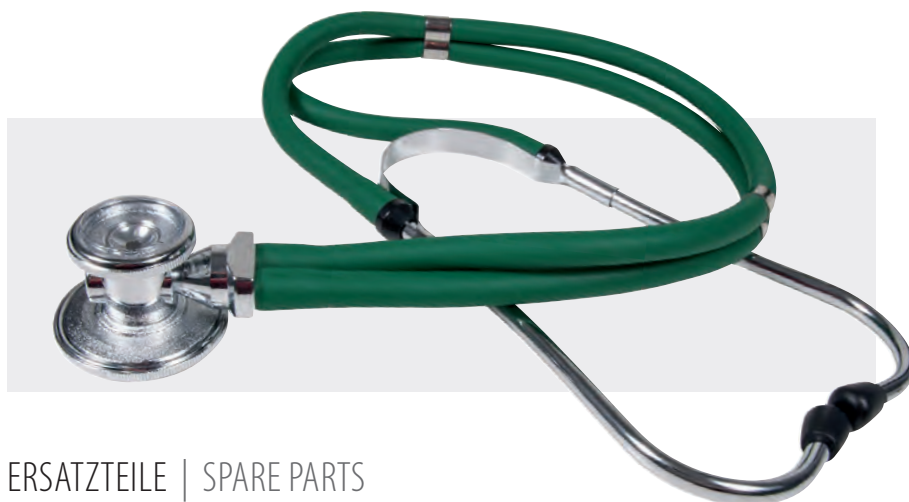
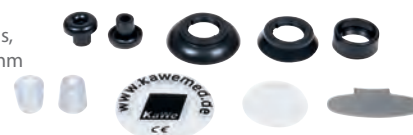
The Top Cardiology stethoscope is the ideal stethoscope for cardiologists. The chest pieces of the KaWe Top Cardiology stethoscope are crafted from solid stainless steel. These quality stethoscopes with excellent acoustics were developed using high-quality craftsmanship and technology. This high-quality stethoscope has a single-tube system with an integrated head piece made of stainless steel; soft ear tips ensure optimal wearer comfort. Each stethoscope comes equipped with two spare pairs of ear tips, one spare diaphragm and a name plate.

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- flat-head stethoscope
- stainless steel
- diaphragm Ø 45 mm
- total length approx. 64 cm
- single-tube system with integrated head piece made of stainless steel
- the ideal stethoscope for cardiologists
- incl. spare parts set
- weight: approx. 210 gr.

139

### LIEFERUMFANG | INCLUDED IN DELIVERY

- Top-Kardiologie Stethoskop
- 2 Ohroliven, schraubbar
- 2 Ohroliven, steckbar
- 1 Ersatzmembrane
- 1 Namensschild
- Top Cardiology stethoscope
- 2 ear tips, screw-on
- 2 ear tips, slide-on
- 1 spare diaphragm
- 1 name plate

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Kirchner COLORSCOP® Plano stethoscopes bring colour into your daily routine. We only use high-quality materials for the manufacture of this stethoscope. Due to its top-rate craftsmanship, this stethoscope is extremely well suited for daily use in doctoral or nursing clinics for (among other things) measuring blood pressure as it fits very well under any blood pressure cuff. The exchangeable diaphragm is reinforced with fibreglass and is therefore especially sensitive. The sound bell is made of metal and therefore guarantees a good resonance because of optimal sound conduction. The available selection of eight different colours almost completely eliminates the possibility of a user mistaking their instrument for that of another.

140

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### Kirchner COLORSCOP® Plano

- das bunte Flachkopf-Stethoskop
- Membrane Ø 43,8 mm
- Gesamtlänge ca. 78 cm
- leichtes Eigengewicht: ca. 99 gr.
- the colourful flat-head stethoscope
- diaphragm Ø 43.8 mm
- total length approx. ca. 78 cm
- lightweight: approx. 99 gr.

Farbbeispiele | Example of available colours

rot   red		REF 06.10100.014
schwarz   black		REF 06.10100.024
blau   blue		REF 06.10100.034
grün   green		REF 06.10100.044

gelb   yellow		REF 06.10100.054
lila   violet		REF 06.10100.094
türkis   turquoise		REF 06.10100.124
pink   pink		REF 06.10100.134

### Ohrlöwen | Ear tips

- schwarz
- Ø 5,0 mm
- VE = 10 St.
- black
- Ø 5.0 mm
- PU = 10 items

 REF 06.91120.021

# Single Stethoskop | Single stethoscope

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The Single flat-head stethoscope is the less expensive alternative to the Kirchner COLORSCOP® Plano. The chest piece is made of aluminium and is therefore especially lightweight.

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- flat-head stethoscope with aluminium chest piece
- diaphragm Ø 43.8 mm
- total length approx. 80 cm
- lightweight:  
approx. 84 gr.

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The Petiphon stethoscope was developed especially for children. The diameter of the bell was conceptualised to better fit the bodies of your small patients and is comfortably warm to the touch thanks to the highly-flexible silicone bell.

Bell type stethoscopes operate (unlike diaphragm stethoscopes) similarly to the human concha and can therefore detect sound very well. By pressing on the bell, the sound is amplified even further. A further advantage is that the sound is captured directly by the bell. With the bells, the user hears sound evenly over the entire diameter of the bell. With diaphragm stethoscopes, one can only hear sound in the very middle.

The robustly-crafted metal chest pieces will serve you dependably for many years. The bell can be easily removed for cleaning, therefore guaranteeing a high hygienic standard for your daily medical practice routine. The double-tube system ensures good acoustics and together with the collapsible head piece, this stethoscope can be easily stowed in any lab coat pocket.

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- bell type stethoscope
- with rotatable ear tips
- flexible, adjustable head piece
- bell Ø 24 mm
- total length approx. 75 cm
- weight: approx. 190 gr.

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- bell type stethoscope with rotatable ear tips
- flexible, adjustable head piece
- bell Ø 49.5 mm
- total length approx. 88 cm
- weight: approx. 240 gr.

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REF 06.95648.001

# Plano/Duo Stethoskope | Plano/Duo stethoscopes

## SCHWESTERN-LEHR-STETHOSKOP | TRAINING STETHOSCOPES FOR NURSES

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Our training stethoscopes for nurses are the Kirchner COLORSCOP® Plano and Duo. The training stethoscopes for nurses consist of two head pieces, a 110 cm-long tube and one flat-headed or dual-headed chest piece. With this instrument, two people can perform an examination at the same time.

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- with 2 head pieces
- diaphragm Ø 43.8 mm
- total length approx. 130 cm
- weight Plano: approx. 210 gr.
- weight Duo: approx. 225 gr.

144

### Schwester-Lehr-Stethoskop Training stethoscope for nurses

- Plano, rot-gelb
- Flachkopf-Stethoskop
- Plano, red-yellow
- flat-head stethoscope

Plano  REF 06.34100.144

- Duo, rot-gelb
- Doppelkopf-Stethoskop
- umschaltbar
- Duo, red-yellow
- dual-head stethoscope
- reversible

Duo  REF 06.34200.144

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#### Ohroliven | Ear tips

- schwarz, Ø 5,0 mm
- VE = 10 St.
- black, Ø 5.0 mm
- PU = 10 items

 REF 06.91120.021

# Pinard-Stethoskope | Pinard stethoscopes

## STETHOSKOP AUS BUCHENHOLZ | STETHOSCOPE MADE OF BEECHWOOD

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The Pinard stethoscope is an obstetric stethoscope that was developed in 1895 by the French doctor and obstetrician Adolphe Pinard. It is made of beechwood and is ideal for midwives and expectant parents and enables one to hear the heartbeat of the unborn child.

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- beechwood
- approx. 17 cm long
- Ø 44 mm
- weight: approx. 45 gr.

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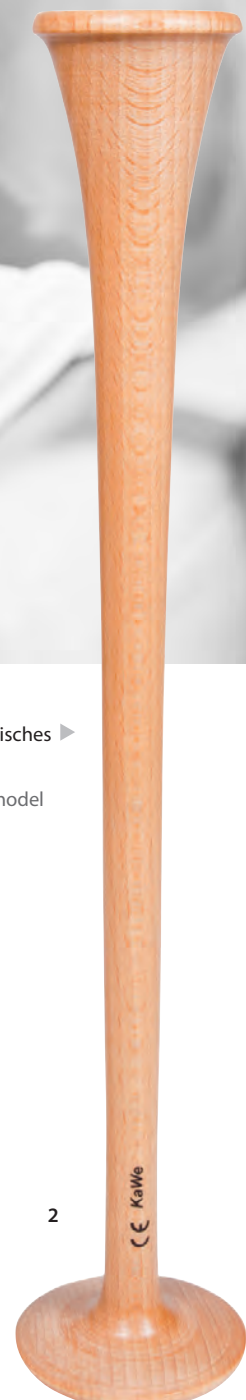
- beechwood
- approx. 33 cm long
- Ø 46 mm
- weight: approx. 65 gr.

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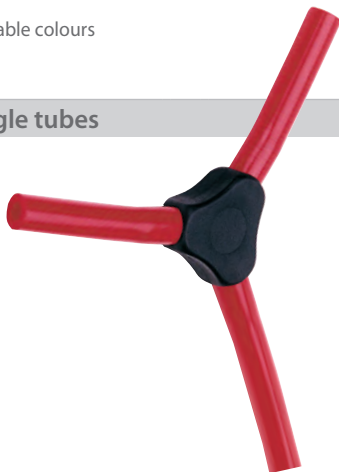
for the Cardiology-line  
soft, screw-on

schwarz, VE = 10 St.  
black, PU = 10 items

REF 06.91220.021

147

## SONSTIGES | OTHER ARTICLES

Y-Schlauchstück  
Y-tube piece

schwarz, aus Kunststoff  
black, made of plastic

für | for REF  
06.10100.014 - 06.10100.134,  
06.22100.014 - 06.22100.134,  
06.34100.144, 06.34200.144

REF 06.99020.021

Namensschild  
Name plate

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suitable for all  
KaWe stethoscopes

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- reflex hammer
- large
- black
- with neurological sensor tip

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- reflex hammer
- large
- blue
- with neurological sensor tip

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- reflex hammer
- large
- grey
- with neurological sensor tip

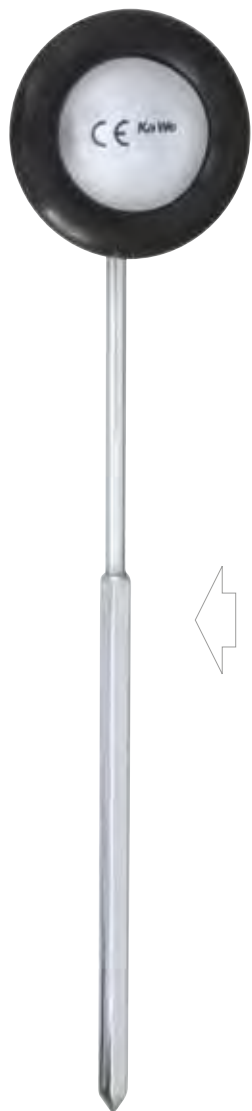
ca. | approx. 22 cm  
ca. | approx. 110 g

REF 07.15204.262

## BABINSKY

## ROSSIER

150

**Babinsky**

- Reflexhammer
- Metallgriff umschraubbar

- reflex hammer
- metal handle, interchangeable with screw fitting

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- reflex hammer
- elastic plastic handle interchangeable with screw fitting

Ø 45 mm  
ca. | approx. 23 cm  
ca. | approx. 70 g

REF 07.05300.001

**Rossier**

- Reflexhammer
- elastischer Kunststoffstiel

- reflex hammer
- elastic plastic handle

Ø 50 mm  
ca. | approx. 24 cm  
ca. | approx. 90 g

REF 07.10324.001

**Rossier**

- Reflexhammer
- elastischer Kunststoffstiel

- reflex hammer
- elastic plastic handle

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- Fassbender combination hammer
- with solid metal handle, with scale setting in cm
- with brush
- with needle

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- reflex hammer
- with metal handle: original model
- heavy
- without brush
- without needle

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- reflex hammer
- with black plastic handle
- heavy
- with brush
- without needle

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- reflex hammer
- with black plastic handle
- light
- with brush
- with needle

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- reflex hammer
- with black plastic handle
- heavy
- with brush
- with needle

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- reflex hammer
- with metal handle
- light
- with brush
- with needle

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- reflex hammer
- with plastic handle
- light
- with brush
- with needle

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KaWe sets high standards for the tuning fork production. Our tuning forks are not manufactured in series, but each one is manually tuned several times ensuring that it is guaranteed to hold its pitch. Each original KaWe tuning fork is labelled according to its tone and corresponding frequency. The specially-selected materials permanently guarantee long tone stability, long-lasting vibration and the sound purity that you are accustomed to. This small classic is simultaneously a precision instrument for your doctoral practice.

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## Tuning forks – indispensable for complete patient examinations

Medicinal tuning forks will forever be associated with names such as Weber and Rinne or Rydel Seiffer. Every carefully-done, complete physical examination of a patient involves the use of a tuning fork.

The graduated tuning fork developed by Rydel Seiffer c 128 Hz/C 64 Hz is a standard in the field of neurology and is also probably the most well-known neurological tuning fork. This calibrated tuning fork has removable dampers with a 1/8 scale graduation and is used to identify a loss of the sense of vibrations, and is used to diagnose polyneuritis which can occur e.g. along with diabetes mellitus. With the screwed-on dampers, the fork vibrates at 64 Hz. Reference marks for exact calibration of the original KaWe tuning fork can be found on the back side of the fork tines. It is in this configuration that the tuning fork is used in vibration examinations. The screwed-on plastic foot allows for a gentle transfer of the vibrations from the tuning fork to the patient. Without the dampers, the instrument vibrates at a frequency of 128 Hz and is used for simple hearing and bone conductivity tests. This tuning fork method for testing sensibility, which was first used in 1903, is still considered to be the most dependable and safe even today.

The Weber experiment serves to identify a laterality of the sense of hearing with the use of a tuning fork. Physiologist Ernst Heinrich Weber (1795–1878) of Leipzig first recorded this in experiment in 1825. Hildesheimer psychologist Heinrich Rinne (1819–1868) amended the test in 1855. Whereas the Weber test serves to compare the sound perception of each ear by means of bone transmission, the Rinne test compares air and bone transmission. Both tests together form a standard test for the examination of hearing damage as well as of the eighth cranial nerve.

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Tuning forks with plastic foot, high-grade steel, nickel-plated

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For medical diagnoses  
e.g. for audiometry  
(air and bone conduction)  
for aesthesiometry  
and other diagnostic purposes

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For audiometry  
(air and bone conduction)  
for aesthesiometry and  
other diagnostic purposes

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## Infusion stand made of high-quality stainless steel

- 5-foot stand for stability with sturdy plastic juncture at the base
- safety hooks for the infusion bags/bottles
- maximum suspension weight per hook: 2 kg
- includes synthetic drip glass
- height is freely adjustable
- quiet Ø 50 mm double-wheeled castors with 2 locking castors, 2 safety castors and 1 antistatic castor
- assembly tool included
- easy assembly
- individually packaged

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REF 09.91111.001

### Tropfglas | Drip glass

bruchsicher,  
aus Kunststoff  
shatter-proof,  
made of  
synthetic  
material

REF 09.91012.001

# Infusionsständer, schwer | Infusion stand, heavy

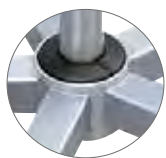
AUS HOCHWERTIGEM EDELSTAHL | MADE OF HIGH-QUALITY STAINLESS STEEL

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The stability of the infusion stand is guaranteed when infusion syringes and pumps with a maximum weight of 5 kg are hung on the solid lower stand post such that their center of gravity is at most 15 cm from the vertical axis of the post. Always ensure that the load is uniformly distributed in order to prevent tipping.

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## For holding Infusion bags and pumps on lower stand post

- 5-foot stand for high stability (solidly-welded metal juncture enables easy cleaning)
- safety hooks for the infusion bags/bottles
- maximum suspension weight per hook: 3 kg
- the total suspension weight of 8 kg must not be exceeded
- greater stability (tilt of 10% when a 2 kg load is hung from one side of the safety hook assembly)
- includes synthetic drip glass
- height is freely adjustable
- quiet double-wheeled castors with 2 locking castors, 2 safety castors and 1 antistatic castor
- assembly tool included
- individually packaged

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## Infusion stand, chrome-plated

- 5-foot stand for stability with sturdy plastic juncture at the base
- safety hooks for the infusion bags/bottles
- maximum suspension weight per hook: 2 kg
- includes synthetic drip glass
- height is freely adjustable

- quiet Ø 50 mm double-wheeled castors with 2 locking castors, 2 safety castors and 1 antistatic castor
- assembly tool included
- easy assembly
- individually packaged

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- minimum height: approx. 120 cm
- maximum height: approx. 210 cm
- Ø of stand foot: approx. 60 cm
- approx. 3.2 kg

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- synthetic leather
- thickness of cushion: 55 mm
- washable, disinfectable
- overall weight: 32 kg
- rated load: 400kg
- moving load: 135kg
- adjustable backrest: 9 latch settings up to approx.+34°
- cushion distance on movable backrest protects fingers
- frame: powder-coated, grey-white
- incl. paper roll holder
- legs with non-marking leveling feet
- Delivered pre-assembled (one piece)

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- synthetic leather
- seat height: 450 – 570 mm
- thickness of cushion: 50 mm
- seat cushion Ø 37 cm
- overall weight: approx. 5.5 kg
- load capacity: 110 kg max.
- safety pneumatic spring with release lever
- the castors are suitable for carpet

**Drehhocker Classic | Swivel stool Classic**

1

1	schwarz   night		REF 09.51302.022
2	blau   blue		REF 09.51302.032
3	grau   grey		REF 09.51302.062

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AUS EDELSTAHL | MADE OF STAINLESS STEEL

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- available with 1 or 2 steps
- load capacity: approx. 200 kg

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**Lights out – Spotlight on!**  
**See clearly with medical examination lights.**

Medical examination lights are indispensable when examining tissue structure. With these lights, you are able to analyze the examination area exactly and therefore effectively make a concrete diagnosis. Medical examination lights provide you with optimal visibility while treating your patients, be it during an operation or otherwise.

KaWe MASTERLIGHT® lamps have been an important part of KaWe's product line for years. Working with a KaWe MASTERLIGHT® medical examination lamp is like working in full daylight. The light can be focused precisely. With the brightness adjustment switch, the lamp can be adapted to any needed lighting situation. Even the smallest colour nuances in the tissue can be seen thus making it easier to make a precise diagnosis and perform safe treatment.

In addition to our well-received halogen models, our KaWe MASTERLIGHT® lamps are also available with visionary LED illumination technology. They offer, in addition to the named advantages of halogen light technology, infrared-free, cool, high-contrast light and a long working life.

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#### Flexible halogen examination light with 5-foot, rolling floor stand made of high-quality stainless steel

- modern, slim design
- precise light spot – easy to use, infinitely adjustable
- supply voltage: 230 Volt
- lamp: 12 V / 35 Watts, with halogen cold light reflector
- reflection angle: 8°
- colour temperature: approx. 3200° Kelvin
- easy and quick assembly
- casters: 2x with brake, 1x anti-static and 2x normal

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for MASTERLIGHT® HL,  
with safety glass protection shield  
lifespan of the HL light: 1,000 h,  
reflector angle of reflection: 8°

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LED bulb: 7 Watts
Lifespan of the LED light
Illumination intensity (at a working distance of 0.4 m)
With universal power plug

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#### Flexible LED examination light with 5-foot, rolling floor stand made of high-quality stainless steel

- modern, slim design
- precise light spot – easy to use, infinitely adjustable
- supply voltage 100 - 240V, 50 - 60 Hz
- LED bulb
- reflection angle: 24°
- colour temperature: approx. 2700° Kelvin
- easy and quick assembly
- with universal power plug
- casters: 2x with brake, 1x anti-static and 2x normal

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for MASTERLIGHT® LED, LED reflector bulb  
angle of reflection: 24°

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- upper part only, 35 W
- halogen bulb
- black
- with gooseneck
- without stand

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- upper part only, 7 W
- LED bulb: 7 Watts (80% less energy compared to 35W)
- black
- with gooseneck
- without stand
- with universal power plug

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used to affix the upper part of the MASTERLIGHT® HL and LED to the wall

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used to affix the upper part of the MASTERLIGHT® HL and LED to table tops with thicknesses of up to 4 cm

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### MASTERLIGHT® examination lamp LED, 7W

**Excellent colour rendering.** With excellent colour rendering values of  $R_a=95$ , the finest colour nuances in tissue can be recognized. The colour spectrum of the wound area appears natural and rich in contrast. Furthermore, this operating light is noticeably less fatiguing for the eyes.

**Faceted multiple-lens system.** The large number of computer-calculated faceted lenses ensures homogeneity and minimal shading in the light field. Separately arranged lens systems, each with one LED module create their own light fields. This enhances the contrast effect and thus makes details in the wound area more discernible.

A maximum light intensity of up to 30,000 lux can be attained.

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#### Features

- with 5 foot, mobile floor stand made of high-quality stainless steel
- electronic on/off switch on the light body
- low power consumption

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3
95
4000 K
0,5° C
50.000 h
Ø 12 cm
7 W
20-80 cm
60 cm
100-240 V
50-60 Hz
5,0 kg

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#### Technical data

Number of LED lamps
Colour rendering index $R_a$
Colour temperature
Temperature increase near the head
Service life of the lamps
Light body
Overall power consumption
Working distance
Height adjustment range
Power supply
Finish: silk matte
Weight (net)

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#### Fixed focus

Illumination intensity at a distance of 0.5 m
Light spot size

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REF 10.71011.002

# MASTERLIGHT®

## UNTERSUCHUNGSLEUCHE LED, 10W | EXAMINATION LAMP LED, 10W

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### MASTERLIGHT® examination lamp LED, 10W

**Excellent colour rendering.** With excellent colour rendering values of  $R_a=95$ , the finest colour nuances in tissue can be recognized. The colour spectrum of the wound area appears natural and rich in contrast. Furthermore, this operating light is noticeably less fatiguing for the eyes.

**Faceted multiple-lens system.** The large number of computer-calculated faceted lenses ensures homogeneity and minimal shading in the light field. Separately arranged lens systems, each with one LED module create their own light fields. This enhances the contrast effect and thus makes details in the wound area more discernible.

A maximum light intensity of up to 60,000 lux can be attained.

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#### Features

- with 5 foot, mobile floor stand made of high-quality stainless steel
- mechanical on/off switch
- electronic brightness control
- low power consumption

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7
95
4000 K
0,5° C
50.000 h
Ø 22 cm
10 W
20-80 cm
60 cm
100-240 V
50-60 Hz
6,0 kg

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### MASTERLIGHT® examination lamp LED, 18W MASTERLIGHT® examination lamp LED, 18W, focus

**Excellent colour rendering.** With excellent colour rendering values of  $R_a=95$ , the finest colour nuances in tissue can be recognized. The colour spectrum of the wound area appears natural and rich in contrast. Furthermore, this operating light is noticeably less fatiguing for the eyes.

**Faceted multiple-lens system.** The large number of computer-calculated faceted lenses ensures homogeneity and minimal shading in the light field. Separately arranged lens systems, each with one LED module create their own light fields. This enhances the contrast effect and thus makes details in the wound area more discernible.

A maximum light intensity of up to 50,000 lux can be attained.

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#### Features

- with 5 foot powder coated mobile stand
- electronic on/off switch on the light body
- electronic brightness control
- low power consumption

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#### Technical data

Number of LED lamps
Colour rendering index $R_a$
Colour temperature
Temperature increase near the head
Electronic brightness control on the light body
Service life of the lamps
Light body
Overall power consumption
Working distance
Height adjustment range
Power supply
Finish: silk matte
Power plug
Weight (net)

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#### Fixed focus

Illumination intensity at a distance of 1 m  
Light spot size

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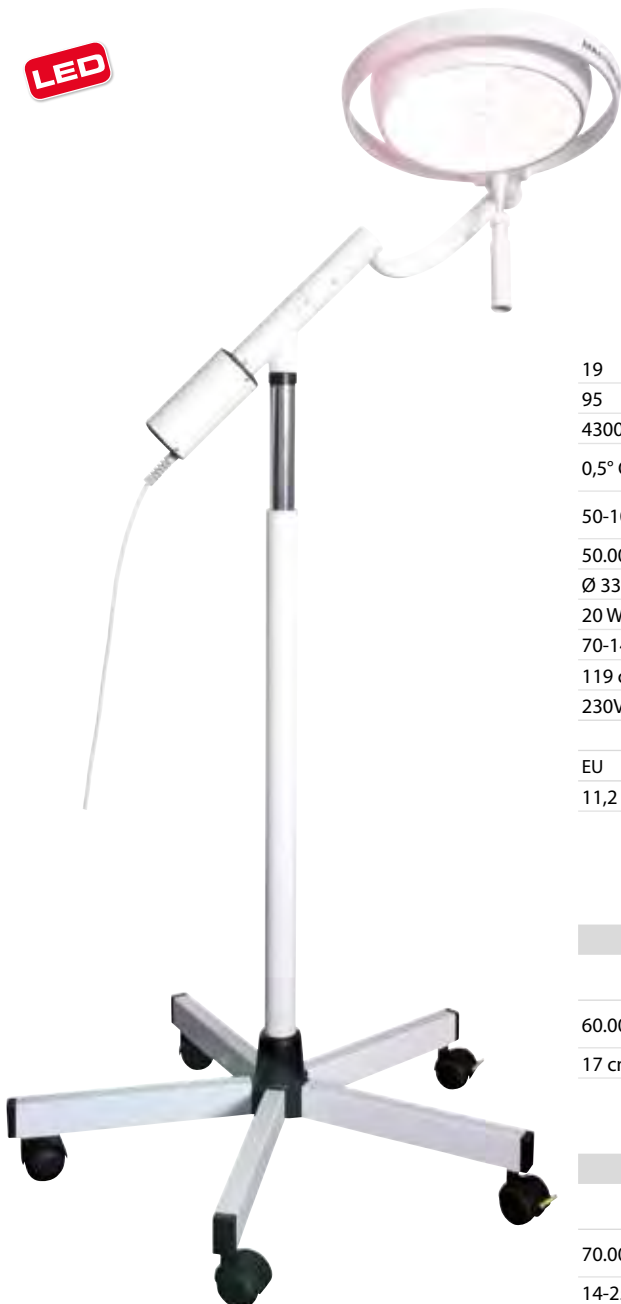
### MASTERLIGHT® examination lamp LED, 20W MASTERLIGHT® examination lamp LED, 20W, focus

**Excellent colour rendering.** With excellent colour rendering values of  $R_a=95$ , the finest colour nuances in tissue can be recognized. The colour spectrum of the wound area appears natural and rich in contrast. Furthermore, this operating light is noticeably less fatiguing for the eyes.

**Faceted multiple-lens system.** The large number of computer-calculated faceted lenses ensures homogeneity and minimal shading in the light field. Separately arranged lens systems, each with one LED module create their own light fields. This enhances the contrast effect and thus makes details in the wound area more discernible.

A maximum light intensity of up to 70,000 lux can be attained.

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#### Technical data

Number of LED bulbs
Colour rendering index $R_a$
Colour temperature
Temperature increase near the head
Electronic brightness control on the light body
Service life of the lamps
Light body
Overall power consumption
Working distance
Height adjustment range
Power supply
Finish: silk matte
Power plug
Weight (net)

1	<b>20W</b>	REF 10.71031.002
---	------------	------------------

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- with 2 cannulae,  
olive-shaped  
and conical,
- with 3 rings

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- stainless steel/plastic
- sterilizable
- with special non-slip surface (allows for better grip)
- individually packaged

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Can be used for people and animals, sterilizable.

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- stainless steel/plastic
- large, 19 cm
- serrated lower edge

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- stainless steel, matte finish
- for the medical ward and emergency medicine

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- straight
- stainless steel, matte finish
- for the medical ward and emergency medicine

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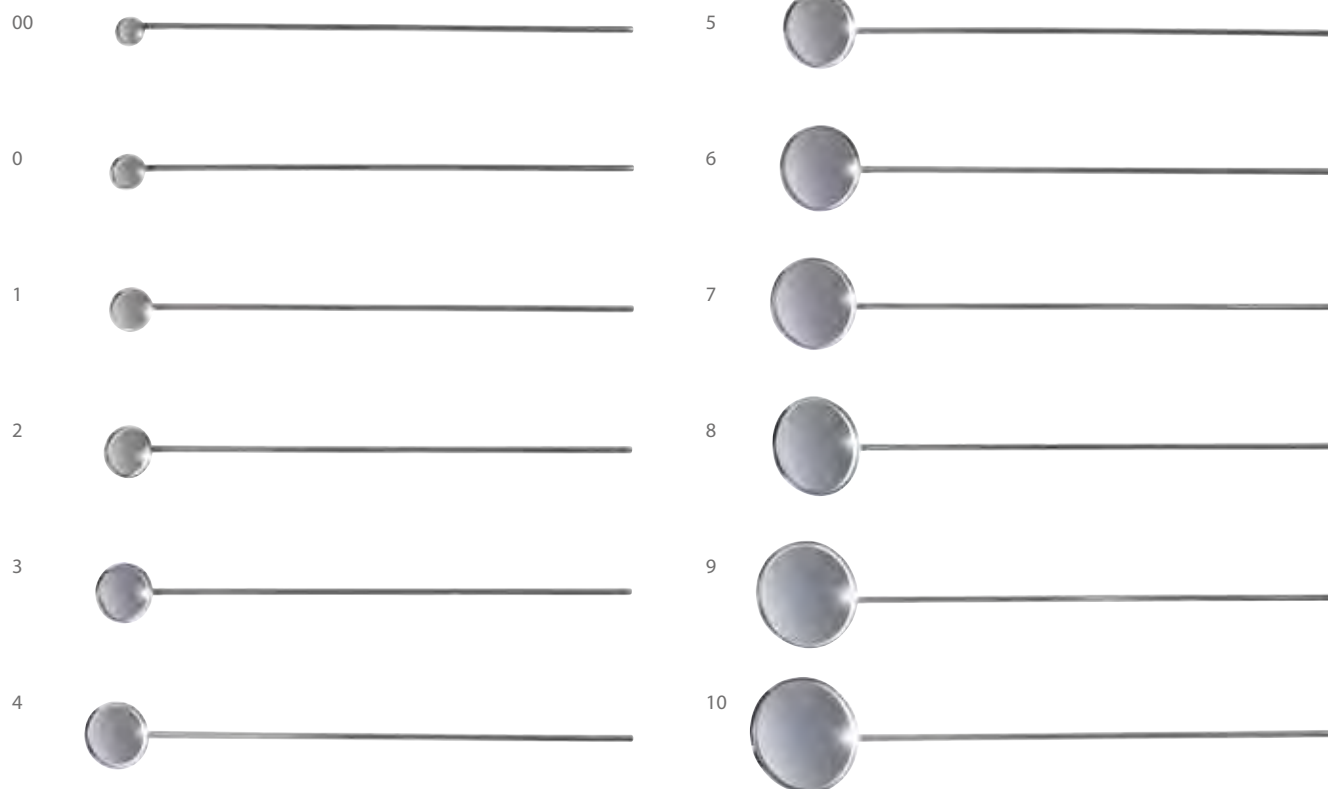
## Schraubgriff | Handle with screw thread

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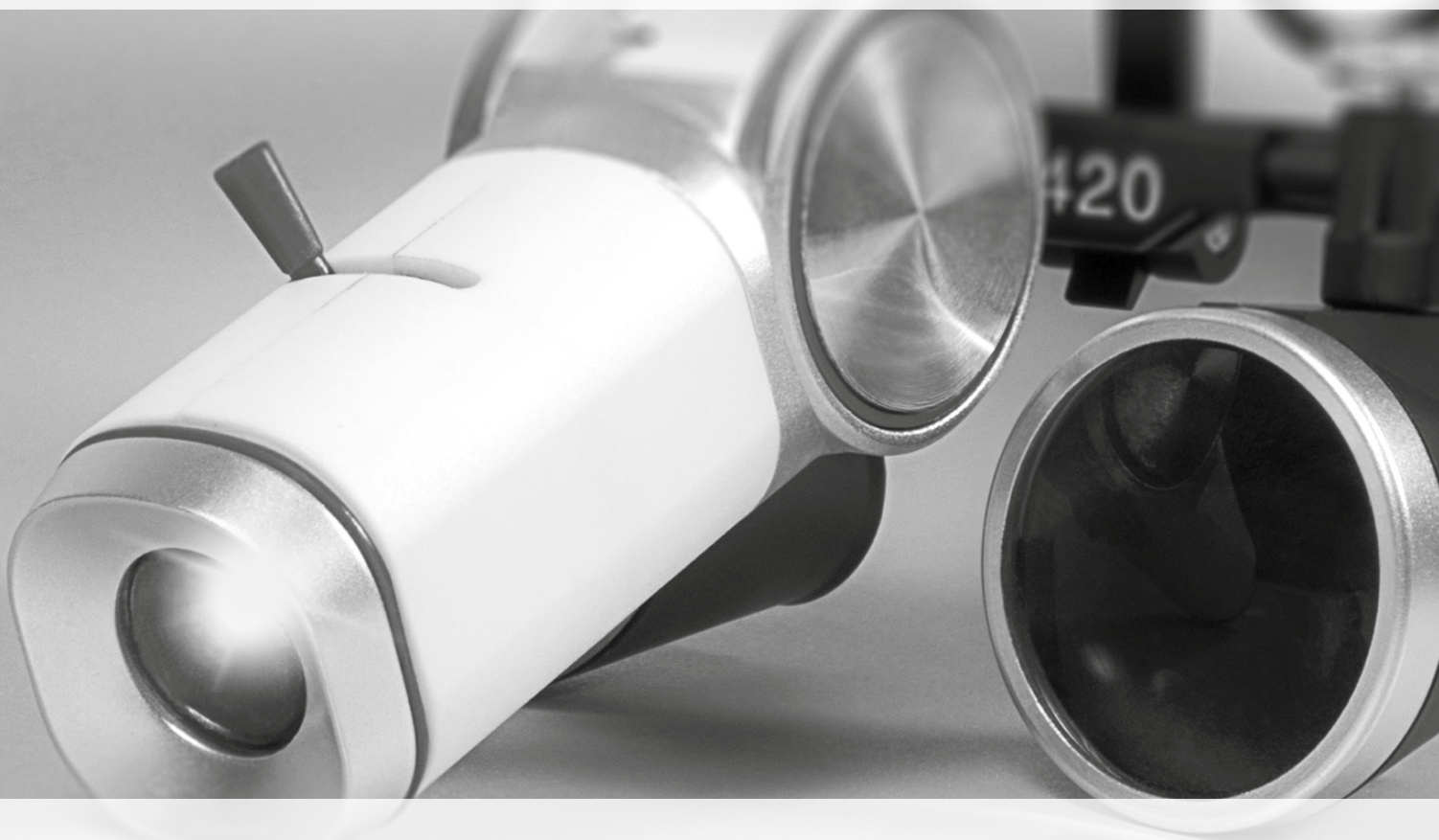
- for laryngeal mirror, made of metal
- PU = 5 items

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### FEATURES:

- Ergonomic and modern design with LED technology
- High-powered LED with white, uniform light for all kinds of diagnostic applications, such as in dentistry, ear-nose-throat medicine, etc.
- Coated optics ensure optimal illumination and less heat development
- Highly-efficient LED, therefore long battery life
- Aluminum cover on the joints keeps the lamp cool through convection and radiation, eliminating the need for additional cooling with noisy fans
- Freely adjustable illumination area size, iris shutter, and illumination angle
- Can be used with binocular magnifying glasses
- Freely and easily adjustable headband thanks to an integrated leveling system. No troubling screws under hoods.
- Replaceable forehead padding
- Very lightweight headlamp: approx. 160 grams without rechargeable battery, approx. 390 grams with rechargeable battery. Optimal comfort is ensured even when used for long periods of time.
- Quick and easy replacement of the rechargeable batteries with the clip/plug system

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## Li-Ion

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Rechargeable battery for headband

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Rechargeable battery with connector cable

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## Technical data

LED life
brightness* adjustable with 5 settings
light spot adjustable at a distance of 250 mm
light spot adjustable at a distance of 500 mm
rechargeable battery
charging time of rechargeable battery for headband
charging time of rechargeable battery for belt
charges - rechargeable battery life
light intensity with a light spot Ø of 55 mm and a working distance of 30 cm with optimal focus
battery life of rechargeable battery for headband
battery life of rechargeable battery for belt
voltage (charger)
Guarantee**
24 months from date of purchase
rechargeable battery voltage
power
weight without rechargeable battery

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– Headlamp only  
– incl. charging plug  
– with rechargeable battery for headband

1	EU	REF 12.89310.112
2	GB	REF 12.89310.122
3	US	REF 12.89310.132

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– Headlamp only  
– incl. charging plug  
– with rechargeable battery for belt incl. connector cable

1	EU	REF 12.89311.212
2	GB	REF 12.89311.222
3	US	REF 12.89311.232

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– Headlamp only  
– incl. charging plug  
– with rechargeable battery for headband  
– with rechargeable battery for belt incl. connector cable

1	EU	REF 12.89312.312
2	GB	REF 12.89312.322
3	US	REF 12.89312.332

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magnification x working distance  
plastic housing with glass optics

1	2,5 x 340 mm	REF 12.89342.342
2	3,5 x 340 mm	REF 12.89343.342
3	2,5 x 420 mm	REF 12.89342.422
4	3,5 x 420 mm	REF 12.89343.422

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### LED-Kopfleuchte H-800 LED headlamp H-800

Kopfleuchte allein, ohne Lade-  
batterie, ohne Ladestecker  
Headlamp only, without rechar-  
geable battery, without power  
plug

Größe | size 13,5 - 21,5 cm

REF 12.89935.002

### Ladebatterie, allein rechargeable battery, separate

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Größe | size 20,5 - 28,5 cm

REF 12.89950.102

### Ladebatterie, allein rechargeable battery, separate

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Größe | size 27,5 - 36,5 cm

REF 12.89950.202

### Verbindungskabel connector cable

zu Ladebatterie für Gürtel  
to rechargeable battery for belt

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Our new product line of »Clar« mirrors is characterized by high quality, optimal handling as well as user-friendliness and is excellently suited for routine diagnostic examinations in your surgery. Whether with battery operation or ac power, KaWe offers you the ability to select according to your individual area of use.

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REF 12.89213.022

# Diagnostikleuchten | Diagnostic lights

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## Cliplight Diagnostikleuchte LED | Cliplight diagnostic light LED

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- made of metal, painted
- with clip contact
- smooth surface
- includes replaceable batteries 1.5 V (type AAA)
- practically packaged in blister pack
- length approx. 13 cm

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- made of metal
- with push button
- includes replaceable batteries 1.5 V (type AAA)

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- plastic head
- with xenon bulb 2.5 V (REF 12.75145.003)
- batteries 1.5 V, mignon (type AA)
- with cloth bag

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Tongue depressor holder for easy attachment of wooden blades to the Cliplight diagnostic lights and diagnostic light, white.

VE = 5 St.   PU = 5 items	REF 12.05290.001
---------------------------	------------------

### Stofftasche, allein | Cloth bag, separate

für KaWe DIALIGHT® XL Diagnostikleuchten  
REF 12.05302.021, 12.05302.231

for KaWe DIALIGHT® XL diagnostic lights  
REF 12.05302.021, 12.05302.231

schwarz   black	REF 01.78401.021
-----------------	------------------

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- blue band, white multi-function clip
- quick single-hand operation
- elastic tourniquet band

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- violet band with child-friendly design, white multi-function clip
- quick single-hand operation
- elastic tourniquet band

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- for veterinarians
- elastic cord, blue, exchangeable

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- C.F.C.-free, 500 ml
- prevents rubber, latex and plastic instruments from sticking to the mucous membrane
- protects against incrustation
- suitable for the lubrication of metallic, surgical instruments
- prevents rust build-up

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- made of synthetic material
- for alcohol-based disinfectants
- one-hand operation
- non-slip base, Ø approx. 10 cm
- height: approx. 7.5 cm

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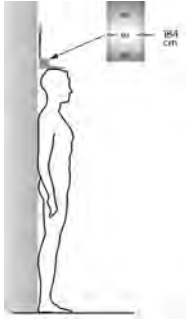
- housing and measuring tape made of plastic
- cm/inch scale
- ø approx. 5 cm
- PU = 5 items

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- suitable for ECG paper from different manufacturers
- length: approx. 211 mm x width: approx. 86 mm
- PU = 5 items

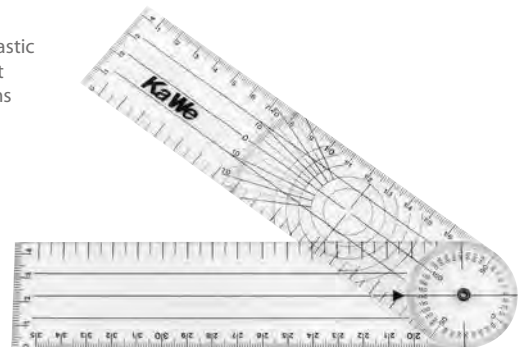
transparent | transparent

REF 12.20400.001

## Winkelmesser Goniometer | Goniometer

- aus Kunststoff
- transparent
- VE = 5 St.

- made of plastic
- transparent
- PU = 5 items

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Westergren dispettes for blood sedimentation.  
Advantages: made of unbreakable polystyrene, optimal safety – eliminates the risk of infection, high measuring accuracy, simple and convenient operation.  
Dispettes, graduated according to DIN 12 845 with filling cap

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Plastic dispettes stand for 30 Westergren dispettes  
standard- and inclined/rapid method  
standard method at 90° 60 - 120 minutes  
rapid method at 45° 6 - 9 minutes  
or at 60° 7 - 10 minutes

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**KaWe MedCharge® 4000** is the charging station for the KaWe EUROLIGHT®, KaWe COMBILIGHT®, and the laryngoscope product lines metal handles in small (AA) and medium (C) models, with the exception of the Economy, LED 2.5V and plug-in charging handles. The operation time of a medium-sized rechargeable batteries is approximately 3 hours, whereas the smaller models last approx. 2 hours. This is however, dependent on the type of bulb used.

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### Performance characteristics:

input voltage	100 V - 240 V +/- 10%, 50 - 60 Hz
max. power	1.2 A
efficiency factor	90 - 93 %
chargeable capacity	min. 250 mAh, max. 4500 mAh
input voltage charging station	9 VDC +/- 10 %
output voltage in stand-by	< 9 VDC
rated output voltage	2x 3.6 VDC
rated charging current	2x 250 mA +/- 10 %
charge end power	10 - 30 mA for Li-Ion
electric strength	4 kV
protection class	II
operating temperature	0° C - +40° C
storage temperature	-25° C - +70° C

### KaWe MedCharge® 4000

- |                                   |                                      |
|-----------------------------------|--------------------------------------|
| – Ladestation                     | – charging station                   |
| – für Ladegriffe 2,5 V / 3,5 V    | – for charging handles 2.5 V / 3.5 V |
| – Intelligente Elektronik         | – intelligent electronics control    |
| – mit wechselbarem Netzstecker    | – with universal power plug          |
| ALL in ONE                        | ALL in ONE                           |
| EU 240 V / GB 240 V / US 110 V    | EU 240 V / GB 240 V / US 110 V       |
| – 2 Adapterhülsen für Griffe (AA) | – 2 adapter sleeves for (AA) handles |
| – inkl. Wandhalterung             | – incl. wall-mounted holder          |

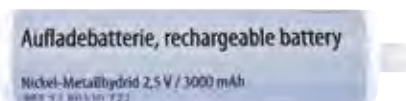
EU / GB / US

REF 12.80005.002

### Ladebatterie (für KaWe MedCharge® 4000)\*

#### Rechargeable battery (for the KaWe MedCharge® 4000)\*

- |  |   |
|--|---|
| <b>Wichtig:</b> Dürfen nicht mit LED-Otoskopgriffen und LED-Laryngoskopgriffen 2,5 V verwendet werden. | <b>Important:</b> can not be used with LED otoscope handles and 2.5 V LED laryngoscope handles. |
| – Ladedauer ca. 14 h   | – charging time approx. 14 h  |
| – Kapazität 3000 mAh   | – capacity 3000 mAh   |

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#### Rechargeable battery (only for the KaWe MedCharge® 4000)\*

- |  |   |
|--|---|
| <b>Vorteile:</b> hohe Lebensdauer, hohe Energiedichte, kein Memory-Effekt, geringe Selbstentladung | <b>Advantages:</b> long service life, high energy density, no memory-effect, minimal self-discharge |
| – Ladedauer ca. 8-10 h   | – charging time approx. 8-10 h  |
| – Kapazität 2200 mAh   | – capacity 2200 mAh   |

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### Flawless optics

The ability to observe images of accurately reproduced structures with both eyes and perform precise work with both hands – magnifying eyeglasses offer all these features. Increasing demands on precision and quality as well as new surgical technologies make magnifying eyeglasses an indispensable aid in the field of medicine. They are to be used in the dental, dermatological, otholaryngological, ophthalmological fields and in general medicine, in neurosurgery, during microsurgical operations, in veterinary medicine and also during other types of medical examinations.

KaWe magnifying eyeglasses ensure precise vision and an individually adjustable ergonomic operating position. These user-friendly designs are available as binocular magnifying glasses (BS 2.3) and safety glasses with classical spectacle frames (BC 2.3). Coated optical components and the rough structure of the optical tube reduce diffused light to an indiscernible level (less than 1%). All models are equipped with a finely adjustable duplex joint with an optimally matched mechanism of CNC milled solid material. The technology of KaWe magnifying eyeglasses is based on state-of-the-art BiTech optics that makes achromatic lenses dispensable. This technology makes the magnifying eyeglasses lighter and less expensive. It goes without saying that all KaWe models ensure a perfect fit, stay securely in place and have optics that are both dust and splash proof.

The KaWe alcohol-based cleaning spray ensures optimal care. It covers the optical surface of the lens with a protective film and ensures permanent protection.

# Binokularlupen | Binocular magnifying glasses

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## Features BC 2,3 / BS 2,3

- with HRI oculars
- The oculars can be individually adjusted to fit any pupillary distance (PD)
- Optics with 2.3 times magnification
- The optics are dustproof and splash-proof
- The state-of-the-art BiTech optics makes achromatic lenses unnecessary  
This technology makes the magnifying eyeglasses lighter and less expensive
- Coated optical components and the rough structure of the optical tube (MT9019, REF 33, VDI3400) reduce diffused light to an indiscernible level (less than 1%).
- The preadjusted angle of vision enables an operating distance of 340 mm and a field of vision of 110 mm

# Binokularlupen | Binocular magnifying glasses

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The KaWe BC 2.3 Binocular Magnifying Glasses are the classical model of KaWe binoculars. The metal eyeglass frames fit perfectly thanks to spring hinges and exceed all expectations due to their simple but precise mechanical design. The length of the anti-slip frames can be freely adjusted within a range of 125 to 150 mm.

- Anti-Rutsch-Bügel
- längenverstellbarer Bügel
- Einstellung Pupillendistanz
- Anti-slip ear pieces
- Length-adjustable ear pieces
- Adjustable pupillary distance

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Included:

- Cleaning spray 25 ml (alcohol and solvent-free, anti-static)
- Microfibre cloth
- Exclusive spectacle case

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- These oculars can be adjusted thanks to the finely-adjustable duplex joint and optimally-matched mechanics made of CNC-cut solid material. The tilt angle can be set easily and securely. This enables the wearer to assume a comfortable and fatigue-free work posture.
- for more features see (»194)

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Included:

- Cleaning spray 25 ml (alcohol and solvent-free, anti-static)
- Microfibre cloth
- Exclusive spectacle case
- Securing strap

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KaWe thermometers are especially well-suited for use in medical clinics because they are built to withstand the requirements of such environments: with the best technologically-achievable measurement accuracy ( $\pm 0.1$  °C), **the extremely quick measurement response time of only 10 seconds** ensures that if the patient moves, which often occurs in clinical situations, this movement has a minimal effect on the variability of the measurement. Repeat measurements can therefore be avoided, even during difficult clinical conditions. All KaWe thermometers fulfill the requirements of EN 12470-3, ISO 80601-2-56 and ISO EN 10993 standards and are intended for use in hospitals and healthcare facilities.

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## Features / technical data:

- Type: digital thermometer, standard and flexible
- High-precision end value
- Measurement range: 32.0°C – 42.9°C
- Measurement accuracy:  $\pm 0.1^\circ\text{C}$  within 35.5°C and 42.0°C
- Display: large, easy to read liquid crystal display (LCD)
- Displays the last measured value
- Long battery life of  $\geq 200$
- Classification: Type BF, Medical Products Class IIa
- Fever alarm signal
- Quick measurement (10-second response time)
- Compliant with standards: EN 12470-3, ISO 80601-2-56, ISO EN 10993
- Signal tone upon completion of measurement

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Digital thermometer, standard  
in transparent storage box

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For patients that may not be able to remain still, we offer a thermometer with a flexible measuring tip that keeps the patient safe from injury.

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Digital thermometer, flexible  
in transparent storage box

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## Digital thermometer, infrared

When using the KaWe infrared ear thermometer, the temperature is measured on the eardrum, which due to its proximity to the Arteria tympani, reflects the core body temperature almost without delay. This means that at this location, rising and sinking temperatures are measured immediately. **The measuring time of the KaWe Temp® 30IR itself is short, and takes only approx. 1 second.** All KaWe thermometers fulfill the requirements of EN 12470-3, ISO 80601-2-56 and ISO EN 10993 standards and are intended for use in hospitals and healthcare facilities.

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### Features / technical data:

- Type: infrared measurement in auditory canal
- Measurement range: in the ear: 28.0°C – 43.0°C
- Standardized measurement accuracy:  $\pm 0.2^\circ\text{C}$
- Clinical repeatability:  $0.19^\circ\text{C}$
- Duration of measurement: approx. 1 second
- Battery life: approx. 1 year / 3,000 measurements
- Classification: Type BF, Medical Products Class IIa
- Fever alarm signal
- Stores up to 10 values
- With biocompatible, easily-applicable and removable probe covers

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KIRCHNER & WILHELM GmbH + Co KG, Asperg

# KaWe Germany – KIRCHNER & WILHELM GmbH + Co. KG

## General Terms and Conditions of Sale for Transactions with Purchasers in Trade and Industry

### 1. General Terms and Effectiveness

All our deliveries, services and offers to purchasers in trade and industry are exclusively based on these General Terms and Conditions of Sale. These terms shall apply even if they are not expressly declared a second time and shall be binding for all future business transactions. Any deviating terms and conditions shall become legally binding only upon our written confirmation. We herewith expressly and finally contradict any of our Purchasers' terms and conditions of sale that deviate from ours. They will not be recognized even if after receiving them, we do not revoke them once more.

### 2. Offers and Orders

Our offers are subject to prior sale and not binding. Orders, price agreements and all other additional agreements shall become legally binding for us only when confirmed in writing or after the presenting of an invoice. Any documentation accompanying our offers shall be valid only with the restriction that we reserve the right to make changes in the course of further technical developments and depending on the economic market situation. If after contract conclusion the Purchaser's financial circumstances show a marked deterioration which becomes evident only after the finalization of the contract jeopardizing our entitlement, and the agreed terms and conditions of sale, we shall be entitled to repudiate the purchase contract.

Any agreements entered into with our representatives shall be valid upon our written confirmation only. Any subsequent changes or agreements will require our written confirmation in order to become valid.

### 3. Prices

Our prices offered or agreed upon shall be net plus statutory German V.A.T (MWST). If no express price agreement has taken place, prices valid on the delivery date shall apply. If market or production conditions change on the delivery date of the ordered goods, due to circumstances beyond our control, the prices valid on the date of delivery – e.g. the market prices – shall apply. This shall concern business transactions with non-traders only if delivery is to take place later than 4 months after contract conclusion or within the scope of a continuous obligation.

### 4. Time of Delivery and Performance

Delivery dates quoted by us are not binding unless definite fixed dates have been agreed upon in writing. Partial deliveries and partial performances are permissible. If no other terms and conditions have been agreed, the entire quantities of call orders must be accepted within 6 months of contract conclusion. Thereupon and upon prior announcement we shall be entitled to supply any of our warehoused stored goods. In case of delays in delivery and performance due to force majeure events, or events beyond the scope of our responsibility, we shall be entitled to postpone any delivery or performance as long as the force majeure event lasts plus a reasonable start-up period. If the disruption continues for longer than one month, the Purchaser shall be entitled, after a suitable grace period, to withdraw from contract as far as any not yet delivered parts are concerned.

### 5. Delay, Impossibility of Performance

Delays in agreed lead-times due to any force majeure events may not be reasons for damage claims or contract withdrawal.

### 6. Delivery

All shipments shall take place at the Purchaser's risk, as soon as they have left our factory premises, and irrespective as to who bears the freight costs or who carries out the transport. If we carry out the transport, we shall be liable for any damage or injuries caused by ourselves and for any damage or injuries caused by our skilled workers. If transport is carried out by any third parties we shall only be liable for damage or injuries caused during selection, and only for the value of the damaged goods, and only if the Purchaser submits its claims within 5 days in writing. In order to be able to claim damages, any damage of goods incurred during transport must be certified by Railway, Post Office, or Carrier officials when the goods are taken over from any of them.

### 7. Insurance

KaWe shall take out a transport insurance policy for all deliveries within Germany and abroad.

### 8. Defects and Complaints

Complaints due to defects must be submitted to us in writing. Any delivered goods must be checked immediately upon arrival at destination. Delivery shall be regarded as accepted if within 8 days after arrival of the goods at the destination, no written complaint of defects has been submitted. For any hidden defects, the statutory period of limitation shall apply.

In case of justified complaints, non-defective goods shall be supplied once warranty rights have been excluded. Warranty shall be granted for 2 years, which does not exclude the right of damages for non-compliance with assured properties.

### 9. Reservation of Proprietary Rights

For goods delivered by us, we reserve an ownership right as long as there are any outstanding liabilities due to unpaid invoices and unpaid incidental expenses. Incidental expenses shall also be those incurred due to any delay on the Purchaser's side. In case of several liabilities or open account, reservation of proprietary rights serve as collateral security for the settlement of our accounts without regard to whether some goods deliveries have already been paid or not. When receiving checks or bills of exchange we reserve the proprietary rights in the supplied goods until the checks or the bills of exchange have been discharged and honoured. The Purchaser may resell any goods supplied in the ordinary course of business. In so doing it will be its duty to secure our rights when reselling the secured goods to any third party. Any accounts receivable the Purchaser is entitled to out of the resale, the Purchaser shall assign to us even now to the full extent of the title we have in the sold goods, by way of a collateral security, or at least an amount equalling the price agreed upon between us and the Purchaser for the respectively used or sold reserved goods, without requiring any special agreement in each case.

We agree, upon the Purchaser's request, at our own discretion, to release any collateral securities which we are entitled to do, in as much as the amount exceeds the liabilities to be secured by more than 10%.

The Purchaser shall be entitled to collect the assigned accounts receivable for us in a fiduciary capacity for as long as they duly discharge their assigned payment obligations towards us. The Purchaser shall be obliged to forward us any incoming amounts until all our amounts receivable have been paid. Upon our request, the Purchaser shall deliver to us – at any time – the addresses of its clients, as well as the figures and invoice amounts of any deliveries. At any time we shall be entitled to inform those third parties about the assignment. The Purchaser shall not be entitled to pawn the reserved "goods", or pledge them as collateral security. The Purchaser shall be obliged without delay to inform us by telegram or fax of any third party's execution enforcement imposed on the reserved goods or on any collateral securities which we are entitled to.

We shall be entitled and shall herewith be empowered to take possession of the reserved goods without having recourse to any courts if the Purchaser is in delay of payments due. This way of taking possession shall not be deemed as withdrawal from contract. The Purchaser shall be liable for the remaining monetary deficiency once the withdrawn goods have been turned to account.

### 10. Payment Terms

Our invoices shall be payable as follows:

3% discount in case of advance payment and collection by the Bank, 2% discount within 14 days after the invoice date. Net 30 days after the invoice date.

Special payment terms for special productions.

If payment by bill of exchange has been agreed, all charges for bills of exchange and discounts will have to be borne by the Purchaser.

If the payment term is not observed, statutory interest for delay shall be charged. Immediate payment of all accounts payable shall be imposed if due to delay in payment of (a partial) amount, summary proceedings for recovery of debt will be imposed by a court. Payment shall be deemed to have taken place as soon as it is at our disposal. In case of checks, payment shall be deemed to have taken place when the check has been honoured. The acceptance of a bill of exchange takes place upon special payment agreements. Payments by bills of exchange do not count as cash payments and do not entitle the Purchasers to claim any discounts.

Purchasers who are not our regular customers or who place an initial order with us shall receive our goods against advance payment.

### 11. Very Small Orders

For orders within Germany below EUR 100.– net order value

a processing fee of EUR 15.– will be charged.

For orders within the EU countries below EUR 500.– net order value

a processing fee of EUR 20.– will be charged.

For orders from NON-EU countries below EUR 1,000.– net order value

a processing fee of EUR 30.– will be charged.

### 12. Limitation of Liability

Excluded shall be any compensation for defects in our goods deliveries and performance including consequential harm, abortive retroactive performance or violation of any other contractual or preliminary contractual obligations (indebtedness when entering into the contract) of any kind irrespective of the legal reason and notwithstanding the above paragraphs 5 and 8. Also excluded shall be the liability for claims outside this contract, in particular for tortious acts. However, there shall be no exemption from liability:

– if our employees are accused of causing damage intentionally or by gross negligence

– if due to any non-performance of major contract obligations the purpose of this contract (cardinal duties) is imperilled

– if the liability is based on the warranty for grade and quality of the subject matter.

In all cases indemnity /liability shall be restricted to the replacement of a typical contractual damage.

### 13. Applicable Law, Legal Venue, Place of Performance

These mutual terms and conditions and all agreements within the framework of our business relations shall be governed and construed in accordance with the laws of the Federal Republic of Germany. Place of performance shall be Asperg. Legal Venue for any legal rights and obligations also concerning bills of exchange and checks shall be Ludwigsburg, provided Purchasers are fully qualified business men, or legal persons under public law, without any General Legal Venue.

### 14. Severability of Contract Provisions, Storing of Computer Data.

If any provision of these business terms or any condition or provision of any other agreements should be or become void or unenforceable, the validity of the remaining provisions shall remain unaffected.

We shall store the data within the scope of our mutual business relationship pursuant to the German Federal Data Protective Law. (BDSG)

### 15. Use of Graphical Material and KaWe Logos

With regard to the use of the KaWe graphical material and logos please refer to our general terms of use at [www.kawemed.de](http://www.kawemed.de).

### 16. Final Provision

In addition, the delivery and payment terms of the current price-list shall apply.

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NOTIZEN | NOTES

A series of horizontal dotted lines for writing notes.

A series of horizontal dotted lines for writing notes.

© Copyright 2018 by KaWe. Printed in Germany.

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All rights reserved. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form by means, electronic, mechanical, photocopying or otherwise, without prior written permission of KIRCHNER & WILHELM GmbH + Co. KG.

No liability for printer's error and colour deviations.

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KIRCHNER & WILHELM GmbH + Co KG, Asperg, Germany